*No. 1 choice  
of hospitals  
& mothers*

# Freestyle™

- EN** Instructions for use
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing

**EN** Parts List  
**DE** Teileliste  
**FR** Liste des pièces  
**IT** Elenco componenti  
**NL** Onderdelenlijst

**EN** Parts List  
**DE** Teileliste  
**FR** Liste des pièces  
**IT** Elenco componenti  
**NL** Onderdelenlijst

**a** 810.7084

**EN** PersonalFit™ Breastshield 24 mm  
**DE** PersonalFit™ Brusthaube 24 mm  
**FR** PersonalFit™ Téterelle 24 mm  
**IT** Coppa per il seno PersonalFit™ 24 mm  
**NL** PersonalFit™ Borstschild 24 mm

**b** **c** **d** 200.1513

**EN** Freestyle Connector  
**DE** Freestyle Verbindungsstück  
**FR** Connecteur Freestyle  
**IT** Connettore Freestyle  
**NL** Freestyle Connector

**e** 200.1524

**EN** Freestyle Tubing  
**DE** Freestyle Schlauch  
**FR** Tuyau Freestyle  
**IT** Tubicino Freestyle  
**NL** Freestyle Slang

**f** 811.0021

**EN** Bottle 150 ml  
**DE** Flasche 150 ml  
**FR** Biberon 150 ml  
**IT** Poppatoio 150 ml  
**NL** Melkflesje 150 ml

**g** 200.1591

**EN** Motor unit  
**DE** Motoreinheit  
**FR** Bloc-Motor  
**IT** Unità motore  
**NL** Motoreenheid

**h** 810.0473

**EN** Disc  
**DE** Deckeleinlage  
**FR** Disque de transport  
**IT** Disco  
**NL** Deselinleg

**i** 810.7062

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**IT**  
37-47

**NL**  
48-58

Thank you for choosing the Freestyle breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. «Because you care».

## Why Freestyle™?

Freestyle is the best solution, if you have little time and want to pump efficiently and effectively and remain flexible and mobile.

The double pump with the innovative 2-Phase technology encourages milk production and is proven to give you more milk, in less time. Vacuum can be adjusted for your comfort and a cleverly designed memory function enables you to recall your personal pumping rhythm each time you pump. Freestyle is compact and easily transported: it will become an inseparable companion when you are separated from your baby.

### What is 2-Phase Expression®?

Medela has been the first company worldwide to deeply research babies and their feeding behaviour. The results have led to the development of the 2-Phase-Expression-Technology. 2-Phase-Expression is based on the natural sucking rhythm of the baby: Short, stimulating movements (**Stimulation**) are followed by longer, more intensive ones (**Expression**), ensuring an optimal milkflow. Thus, more milk can be expressed in a shorter time – and more comfortably.

## Indications for use

Freestyle is an electric/ battery powered breastpump for supporting breastfeeding mothers. Freestyle is ideal:

- I To help overcome mother or baby breastfeeding problems
  - Support mother of babies with feeding difficulties (cleft lip/palate, prematurity or neurological problems)
  - Enhance flat or inverted nipples
  - Sore and cracked nipples
  - Breast inflammation

# Important safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

Read all instructions before use.

## Product

- | Keep appliance dry and do not place in direct sunlight.
- | Do not get the product wet.
- | Do not place in or drop motor unit into water or other liquid.
- | Do not spray or pour liquid directly onto the pump.
- | Check pump kit components for wear or damage.
- | This is a single user product. Use by more than one person may present a health risk.
- | There are no user-serviceable parts inside the Freestyle. Do not attempt to repair the Freestyle Breastpump yourself.

## Electricity

- | Only use the mains adapter or battery that comes with the Freestyle Breastpump.
- | Make sure the voltage of the mains adapter is compatible with the power source.
- | Plug the mains adapter into Freestyle Breastpump first and then into wall socket.
- | Always unplug electrical devices immediately after using.
- | Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.
- | Keep the mains adapter away from heated surfaces.
- | Do not reach for any electrical device if it has fallen into water. Unplug immediately.

## Use

- | Use the product only for its intended use as described in this instruction manual.
- | Use Medela original accessories only.
- | Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- | The product should never be left unattended when plugged into a power source.
- | Do not use outdoors.
- | Do not use while bathing or showering.
- | Do not use while sleeping or overly drowsy.
- | Do not drive while free-hand pumping.
- | Supervision is necessary when this product is used in the vicinity of children.
- | If problems or pain occur, consult your lactation consultant or physician.
- | Equipment not suitable for use in the presence of explosive atmospheres.
- | Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

- I Portable and mobile radio frequency communications equipment can affect the Freestyle Breastpump.
- I The Freestyle Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

SAVE THIS INSTRUCTION MANUAL

# How to clean Freestyle pump set

Please follow these handling and cleaning instructions. Use drinking water quality only. Disassemble all parts prior to cleaning.

Part	(a)	(b)	(c)	(d)	(e)*	(f)	(h)	(i)	(j)
<b>Clean prior to first and once daily</b>									
Firstly rinse with cold water, second wash thoroughly with hot soapy water, finally rinse. Medela recommends	✓	✓	✓	✓	✓	✓	✓	✓	✓
covering with cold water in a saucepan and boiling out for 5 min.	✓	✓	✓	✓	✓	✓	✓	✓	✓
or washing in top rack of dishwasher	✓	✓	✓	✓	✓	✓	✓	✓	✓
or with Quick Clean Microwave Bag	✓	✓	✓	✓	-	✓	✓	✓	✓
or with b-well Steam Sterilizer	✓	✓	✓	✓	-	✓	✓	✓	✓
<b>Clean immediately after each use</b>									
Firstly rinse with cold water, second wash thoroughly with hot soapy water, finally rinse.	✓	✓	✓	✓	-	✓	✓	✓	✓

\*if it is dirty only

**Note:** Food coloring may discolor components which will not affect performance. Medela can assume no liability for a wrong handling. You can find more information at: [www.medela.com](http://www.medela.com)

# How to prepare your Freestyle

## Control buttons

- 1 On / Off button
- 2 Let-down button
- 3 Timer reset button
- 4 Vacuum increase button (+)
- 5 Memory button

## LCD-Display

- 6 Timer
- 7 Stimulation phase indicator
- 8 AC power indicator
- 9 Battery power indicator
- 10 Expression phase indicator
- 11 Memory «on» indicator

Open the battery compartment by sliding the back panel of the pump down.

Insert the battery into the compartment with the arrows and the note «This Side Down» pointing towards the pump.

Close the battery compartment.

Plug in the mains adapter and charge until the battery power indicator shows four bars and stops flashing (up to 24 h). Pump may be used while charging. Do not unplug the mains adapter until the charging is completed.

## Electrical/ Battery operation

The pump can be mains adapter and battery operated.

**Mains symbol on the display**

The pump is plugged in.

**Battery indicator on the display**

Battery power state

**4 bar-battery indicator on the display**

Full battery, 3h pumping time

**1 bar-battery indicator flashes**

20 min.pumping time only  
Battery has been removed

**Battery indicator flashes**

Empty battery

**"bAtt" flashes on the display**

Charging for first time  
Removal of battery  
(Up to 24h for charging-time)

### Caution:

The battery used in this device may present a risk of fire or chemical burn if mistreated. Do not disassemble, heat or incinerate. Replace battery with Medela part number 200.1512 only. Use of another battery may present a risk of fire or explosion.

When storing the battery for a limited length of time, store within the following temperature range: -20° to 50° C (-4° to 140° F). When storing the units for an extended time, store within the following temperature range 0° to 25° C (32° to 77°F). Dispose of used battery promptly. Keep away from children. Do not disassemble and do not dispose of in fire.

### Note

If the pump has been running for 30 minutes without any manipulation, the Freestyle will automatically switch off.

Avoid removing battery. Removing the battery will reset the battery indicator and requires a recharging time of up to 24 hours to readjust the battery indicator.

# How to assemble your Freestyle pump kit

Insert membrane into connector body.

Snap clear cap onto connector body.

Push the breastshield onto the connector.

Screw connector onto the bottle.

Connect tubing to the breastshields and pump.

For single pumping, place unused tube into tubing dock.

# How to use your Freestyle

Centre your breastshield over your nipple.

**Caution:**

Hold the bottle as on the photo, not on the connector.  
 – Do not tilt or overfill bottle.  
 – Take care not to kink tubing.

Switch on the breastpump by pressing 1.  
 The pump will begin pumping in the Stimulation Phase.

Adjust the vacuum during pumping to your comfort level by pressing 3/4.

Once your milk starts to flow, press 2 to switch to the Expression Phase.  
 After two minutes the pump will automatically switch to the Expression Phase.

**To memorize your personal pattern,** press the button 5 while pumping in the Expression Phase for three seconds, until the display flashes and the pump beeps three times.  
 Finish pumping session as usual.  
**To activate your memorized setting,** start the pump and press 5.

If you wish to use a second Stimulation Phase press 2 again. Once finished pumping switch off the breast pump, and disassemble the pump kit items. Use bottle stand to prevent tip over spillage.

### Timer

The timer automatically starts when the pump is turned on.

If for any reason you want to restart the timer during a pump session, press the button 2 and hold for 3 seconds. This will restart the timer, but Freestyle will continue to pump.

## How to store/thaw breastmilk

Medela recommends using the following guidelines for storing and thawing breastmilk. For Healthy Term Babies.

	Room temperature	Refrigerator	Refrigerator freezer compartment	Freezer
<b>Freshly expressed breastmilk</b>	Do not store at room temperature	3 - 5 days at approx. 4°C	6 months at approx. -16°C	12 months at approx. -18°C
<b>Thawed breastmilk (previously frozen)</b>	Do not store	10 Stunden	Do not refreeze after thawing!	

- I Never fill bottles/bags more than  $\frac{3}{4}$  full to allow space for possible expansion.
- I Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

# Troubleshooting

Trouble	Solution
<b>If there is low or no suction.</b>	All connections of the kit and breastpump are secure. Back cap is firmly attached at three connection points. Hold the bottle on the breastshield not on the connector. Breastshield is assembled correctly and forms a complete seal around the breast. If single pumping, make sure the unused tube is securely connected to the tubing dock. Take care not to kink tubing.
<b>If the pump gets wet</b>	Unplug the mains adapter from the wall socket and turn the pump off. I Make sure not to turn the pump upside down. Display has to face up. I Store the pump for 24 hours in a dry and warm place.  <b>Note:</b> Do not try to speed up the drying process by applying heat or by using any drying equipment. Do not expose the pump to direct sunlight.
<b>«Err» accompanied by 3 beeps</b>	Contact local Medela office or sales point.
<b>Mains adapter symbol flashing and pump does not turn on</b>	Contact local Medela office or sales point.
<b>Battery power indicator flashes and shows four bars</b>	Contact local Medela office or sales point.

# Warranty

In the "International Warranty" booklet you find all information about the warranty.

# Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects in the environment and on public health.

## ....still a question....

### **Am I using the right size breastshield?**

....Every mother is different and needs her personal size, therefore Medela offers five different sizes PersonalFit breastshield.

Answer some questions at

<http://www.medela.com/IW/en/breastfeeding/good-to-know/right-size-of-breastshield.html>  
to know if the actual breastshield is the right one for you.

### **Is it possible to pump hands-free?**

Yes. With Easy Expression you can pump and have your hands free for other activities.

Visit our website

[www.medela.com/IW/en/breastfeeding/good-to-know/tips-hands-free-expression](http://www.medela.com/IW/en/breastfeeding/good-to-know/tips-hands-free-expression),  
there you can find a detailed description of a correct use of Easy Expression.

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IEC 60601-1-2, Table 201

## Electromagnetic Emissions

The Freestyle is intended for use in the electromagnetic environment specified below.  
The customer or the user of the Freestyle should assure that it is used in such environment.

Emission Tests	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Freestyle uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Freestyle is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

**Warning** – The Freestyle should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Freestyle should be observed to verify normal operation in the configuration in which it will be used. The Freestyle was not tested for immunity to electromagnetic disturbances.

- EN** Technical specifications
- DE** Technische Daten
- FR** Caractéristiques techniques
- IT** Caratteristiche tecniche
- NL** Technische gegevens

vacuum (approx.)  
 -20 ... -270 mmHg  
 -3 ... -35 kPa  
 45 ... 120 cpm

In	Out
100 ... 240 V 50/60 Hz	12 V
	1 A / 1,5 A
	alternating current
	direct current

Li-Ion Battery  
 7.4 VDC nominal  
 2000 mAh

122 x 90 x 58 mm

370 g

270 g

Operation  
 Betrieb  
 En Service  
 Utilizzo  
 Gebruik

Transport / Storage  
 Transport / Lagerung  
 Transport / Stockage  
 Transport / Stoccaggio  
 Transport / Opslag

Operation  
 Betrieb  
 En Service  
 Utilizzo  
 Gebruik

Transport / Storage  
 Transport / Lagerung  
 Transport / Stockage  
 Transport / Stoccaggio  
 Transport / Opslag

kPa

Protection class II  
 Type BF

ISO 9001  
 ISO 13485  
 CE (93/42/EEC)

Please see accompanying papers  
 Begleitpapiere beachten  
 Respecter les consignes des documents  
 d'accompagnement  
 Consultare la documentazione allegata  
 Zie bijgevoegde documenten

Must not be disposed of together with household  
 refuse  
 Darf nicht zusammen mit Hausmüll entsorgt werden  
 Ne doit pas être éliminé avec les ordures ménagères  
 Non deve essere smaltito con i rifiuti domestici  
 Mag niet samen met huisvuil weggegooid worden

Tested according to  
 IEC 60601-1/A2:1995

Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
[www.medela.com](http://www.medela.com)

#### International Sales

Medela AG  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)41 769 51 51  
Fax +41 (0)41 769 51 00  
[ism@medela.ch](mailto:ism@medela.ch)  
[www.medela.com](http://www.medela.com)

#### Australia

Medela Pty Ltd, Medical Technology  
3 Arco Lane, Heatherton  
Vic 3202  
Australia  
Phone +61 (0) 3 9552 8600  
Fax +61 (0) 3 9552 8699  
[contact@medela.com.au](mailto:contact@medela.com.au)  
[www.medela.com.au](http://www.medela.com.au)

#### France

Medela France Sarl  
14, rue de la Butte Cordière  
91154 Etampes cedex  
France  
Phone +33 (0)1 69 16 10 30  
Fax +33 (0)1 69 16 10 32  
[info@medela.fr](mailto:info@medela.fr)  
[www.medela.fr](http://www.medela.fr)

#### Germany & Austria

Medela Medizintechnik  
GmbH & Co. Handels KG  
Postfach 1148  
85386 Eching  
Germany  
Phone +49 (0)89 31 97 59-0  
Fax +49 (0)89 31 97 59 99  
[info@medela.de](mailto:info@medela.de)  
[www.medela.de](http://www.medela.de)

#### Italy

Medela Italia Srl  
Via Turrini, 19 - Loc. Bargellino  
40012 Calderara di Reno (BO)  
Italy  
Phone + 39 051 72 76 88  
Fax + 39 051 72 76 89  
[info@medela.it](mailto:info@medela.it)  
[www.medela.it](http://www.medela.it)

#### Netherlands & Belgium

Medela Benelux BV  
Uilenwaard 31  
5236 WB 's-Hertogenbosch  
Netherlands  
Phone +31 73 690 40 40  
Fax +31 73 690 40 44  
[info@medela.nl](mailto:info@medela.nl)  
[info@medela.be](mailto:info@medela.be)  
[www.medela.nl](http://www.medela.nl)  
[www.medela.be](http://www.medela.be)

#### Switzerland

Medela AG, Medical Technology  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)848 633 352  
Fax +41 (0)41 769 51 00  
[contact@medela.ch](mailto:contact@medela.ch)  
[www.medela.ch](http://www.medela.ch)

#### United Kingdom

Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
United Kingdom  
Phone +44 870 950 5994  
Fax +44 870 389 2233  
[info@medela.co.uk](mailto:info@medela.co.uk)  
[www.medela.co.uk](http://www.medela.co.uk)