Silicone Breastmilk Collector

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EN Breastmilk Collector
Instructions for use

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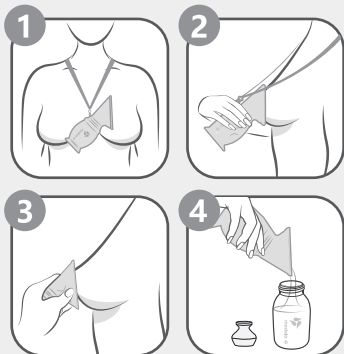
Intended use:

When the mother is breastfeeding or pumping on one side, Medela breastmilk collector can help to collect the spilled breast milk on the other side.

How to use:

1 Before breastfeeding or pumping, attach the lanyard to the breast milk collector and then wear it around your neck to prevent accidental spills during use.

2 Squeeze the left and right sides of the breast milk collector and firmly place the shield against your breast to form an airtight seal. You can then start breastfeeding or pumping.

3 Once breast milk collector is full or you have stopped pumping / feeding, hold on the collector and gently squeeze the shield to remove the collector.

4 After finishing, put the collector on a table, and push down a little, then the suction base will firmly stick to the table surface.

5 When picking it up, gently lift up the tab on the collector's suction base.

6 When you need to store breastmilk for a short time, you can use the silicone leak-proof plug and cap, and store it at room temperature or in the refrigerator.

When you need to store it for a longer period, pour your breastmilk into a Medela storage bag or storage bottle.

Cleaning and maintenance:

- Before the first use, thoroughly rinse and clean the collector. Place all parts except strap into a pan, and fill with cold water (distilled water is recommended). Bring the water to boil and allow it to continue boiling for approximately three minutes. Drain off the water and dry the parts.
- Remove the lanyard from the collector before cleaning. Do not clean with harsh chemicals.
- Always wash all parts immediately after each use. Parts can be washed with a mild cleaning solution, rinsed thoroughly in cold water.
- Dishwasher safe on top rack only.
- Safe to sterilize in Medela Microwave bag.
- Check the collector before each use, and do not use it if there is damage or wear. Replace it right away.
- The collector is made of silicone, which is a material that easily oxidized, so the material will turn yellow after using for a long time. We recommend to replace it every three months.
- Make sure the product is not placed near any sharp objects.

Cautions:

- When using the silicone leak-proof plug, make sure that it is inserted tightly in order to prevent leakage.
- During use, make sure that the collector doesn't press on the milk ducts of the nipple.
- Breastmilk collector may fall due to suction, capacity, skin humidity, etc. Please take precautions in advance.
- Keep the product and accessories away from babies and children.
- The product and its package are not toys, so do not allow babies and children to play with them.
- The lanyard is solely designed and intended for wearing by adults. Do not place it around a baby's neck.
- Never leave the babies and children unattended with the lanyard.
- Do not let the baby chew on the product or its accessories.
- Discard the product if it is damaged.

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