Magic InBra™ Rental

Wearable Breast Pump

Magic InBra™ de location

Tire-lait portable

Electric breast pump
Instructions for use

ENGLISH
2-44

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FRANÇAIS
45-86

The Medela Family Pump Control app offers you the ability to control your breast pump remotely and view the active pumping session information. The app is free and not mandatory for the use of your breast pump. Do not attempt to pair the pump with the app during the pumping session. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Medela AG is under license.

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1. Important safety information

 **Please read all instructions before using this product. Keep these instructions for future reference.**

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Warnings identify all instructions that are important for safety. Failure to observe these instructions can lead to personal injury or damage to the product. The following symbols and signal words show the significance of the warnings:

WARNING

Can lead to serious injury or death.

CAUTION

Can lead to minor injury.

NOTICE

Can lead to material damage (not related to personal injury).

Information

Useful or important information that is not related to safety.

When using electrical products, especially when children are present, basic safety precautions must always be followed.

WARNING

To avoid fire, electric shock, or serious burns:

- Never clean the breast pump during charging.
- Do not place the breast pump charging where it can fall or be pulled into water.

To avoid health risks and reduce the risk of severe injury:

- Use the breast pump only for its intended purpose as described in these instructions for use.
- Magic InBra™ Accessory set is a single-user accessory set for Magic InBra™ Rental breast pump. Use by more than one woman may endanger the health of mother or baby.
- Pregnant women should not use the breast pump, as pumping can induce contractions or premature labor.
- If you suffer from any viral or infectious disease, please be aware that pumping breast milk will not reduce or remove the risk of transmitting the disease to your baby through your breast milk. Please seek medical advice from a licensed healthcare professional.
- If you have an active implanted device that would be affected by magnetic field (e.g. pacemaker or implantable cardioverter defibrillator), consult a licensed health care professional before use.
- Close supervision is necessary when the breast pump or accessories are used in the vicinity of children.
- Clean all parts that come into contact with your breast and breast milk prior to first use and after every use. See **chapter 7** for cleaning instructions.
- Do not use the breast pump while operating a moving vehicle.

CAUTION

To avoid fire or burns:

- The breast pump and accessories are not heat-resistant. Keep away from heated surfaces or open flames.

To avoid health risks and reduce the risk of injury:

- This product cannot be repaired. Do not repair yourself. Do not modify the breast pump.
- Never operate the breast pump
 - if it is not working properly,
 - if it has been dropped, damaged or if mildew has been found on the parts.
 - if it has been dropped into or exposed to water or other liquids.
- Do not use the breast pump while sleeping or overly drowsy.
- Only use Medela recommended parts including the USB power cable with the breast pump.
- Avoid using the pump during sudden pressure changes such as during take-off and landing of aircrafts. In high altitude environments, including pressurized aircraft cabins, the performance of the pump may be affected.

Information

Please follow airline rules for the use of portable electronic devices when flying.

Note: All pictures shown in these instructions are for illustration purpose only. The actual product may vary due to product enhancement. Medela reserves the right to substitute any component or accessory with a replacement of equivalent performance.

Contents

1. Important safety information	3
2. Indications and contraindications	7
2.1 Intended Use / Purpose	7
2.2 Indications for use	7
2.3 Contraindications	7
3. Product description	7
3.1 Breast pump parts	8
3.2 Buttons and status lights	8
4. Getting started	9
5. Charging the breast pump	9
6. Disassembling your Accessory set before first use	10
7. Cleaning	11
7.1 Cleaning overview	12
7.2 Washing and disinfecting	13
8. Assembling your breast pump	14
9. Expressing your breast milk	16
9.1 Operating your breast pump	17
9.2 Realigning the nipple or switching the side	19
9.3 Finishing your pumping session	19
10. Pouring milk from the container	20
11. Disassembling your breast pump after pumping	22
12. Maintenance and care	23
12.1 Long-term storage	23
13. Returning your rental breast pump	24
14. Troubleshooting	25
14.1 Vibrations and Notifications	25
14.2 Problems and Solutions	26
15. Checking if you have the correct breast shield size	31
16. Accessories available for purchase	32
17. Warranty	32
18. Disposal	32
19. Meaning of symbols	33
20. International regulations	36
20.1 Electromagnetic compatibility (EMC)	36
20.2 Simplified EU Declaration of Conformity	40
21. Technical specifications	41
22. Cybersecurity information	43

2. Indications and contra indications

2.1 Intended Use / Purpose

The Magic InBra™ Rental breast pump is a powered breast pump to be used by lactating women to express and collect milk from their breasts.

2.2 Indications for use

The Magic InBra™ Rental breast pump is also indicated to support a mother's milk supply to enable the provision of human milk for infants who are not able to directly or effectively breastfeed (e.g. latch-on problems such as flat or inverted nipples). The breast pump removes milk from affected breast/s to alleviate the symptoms of engorgement and milk stasis as well as support the healing process of sore or cracked nipples and mastitis. The Magic InBra™ Rental powered breast pump is intended for multiple users. The breast pump is intended to be used in a home environment.

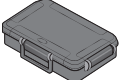
2.3 Contraindications

There are no known contraindications for the Magic InBra™ Rental breast pump.

3. Product description

The Magic InBra™ Rental breast pump is a multi-user electric breast pump with FluidFeel Technology™ and 2-Phase Expression Technology™. This breast pump is engineered for a typical operating life of 500 hours. Actual duration may vary depending on individual usage patterns and environmental conditions. For rental applications, each motor unit is thoroughly inspected, cleaned, disinfected and tested between users to maintain safe and reliable performance.

Your Magic InBra™ Rental breast pump system includes the following:

Motor units	USB power cables	Transport case
		


The Accessory set is a single-user product. The expected operating life of the Accessory set (container, container lid, breast shield, membrane and valves) is six months.

Your Accessory set for use with Magic InBra™ Rental breast pump includes the following:


150ml containers	Yellow valves	Yellow membranes	Breast shields of your choice (18mm, 21mm, 24mm or 27mm)	Grey shield valves
				

For ordering information about accessories see **chapter 16**.

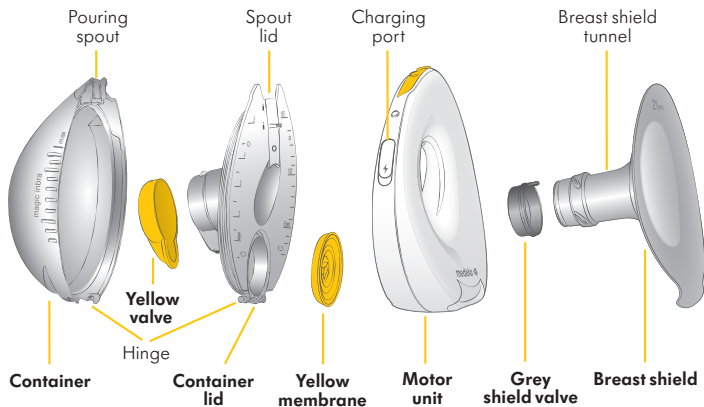3.1 Breast pump parts

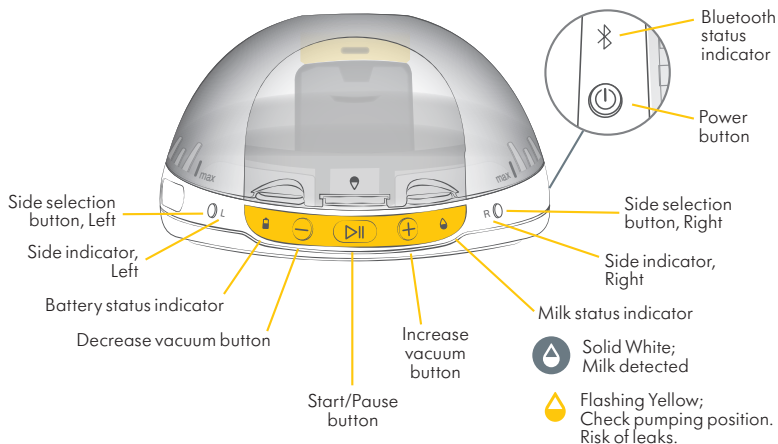3.2 Buttons and status lights

4. Getting started

Before first use, it is important you do the following to ensure optimal performance of the breast pump and safety for you and your baby:

- Charge the breast pump - **chapter 5**.
- Disassemble the Accessory set - **chapter 6**.
- Clean the motor unit by wiping - **chapter 7**.
- Wash and boil the parts (except for the motor unit) - **chapter 7**.
- Assemble the breast pump - **chapter 8**.

5. Charging the breast pump

1. Charge the breast pump for **one hour** prior to first use.

When the power cable is plugged in:

Flashing green; the breast pump is charging

Solid green; the breast pump is fully charged

2. When fully charged, disconnect the power cable and close the charging port.

When the power cable is not plugged in and the breast pump is switched on:

Solid green; the breast pump is charged (enough charge for approximately 150 minutes)

Solid yellow; charging after one/current pumping session advised (approximately 30 minutes left)

Solid red; charge the battery immediately

Two short vibrations and battery status indicator flashing red: Minimum charge level reached. The breast pump will pause automatically. Finish the pumping session and charge the breast pump afterwards.

NOTICE

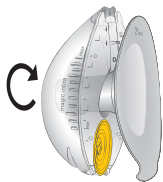
- Never charge the breast pump with a damaged USB power cable.
- Do not charge the breast pump with the milk in the container.
- Do not charge the breast pump while wearing the device in bra.
- Make sure the voltage of the power adaptor is compatible with the power source. See **chapter 21** for technical specifications.

i Information

- The breast pump cannot be turned on or operated when connected to the charger.
- Charging is also possible when the breast pump is assembled.


6. Disassembling your Accessory set before first use

1.

Rotate the container anti-clockwise to detach it.

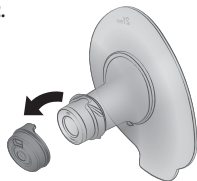2.


Remove the grey shield valve from the breast shield.

3.


Remove the yellow membrane from the container lid.

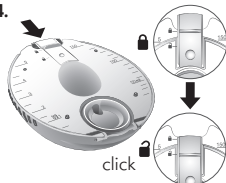4.


Slide the spout lid down to open position.

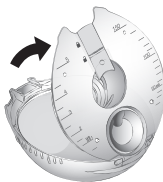5.

Lift the container lid open. This may require some force due to the sealing feature.

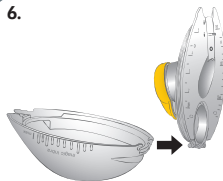6.

Detach the container lid from the hinge.

7.

Remove the yellow valve from the container lid.

7. Cleaning

CAUTION

To keep the pump parts clean and hygienic:

- Disassemble and wash all parts that are exposed to breast milk immediately after use. This will help to remove breast milk residue and prevent growth of bacteria.
- Only use drinking-quality water or bottled water for washing and disinfecting.
- Do not place pump parts directly in the sink to rinse or wash. Use a clean wash basin and a soft sponge used only for infant feeding items.
- Wash your hands thoroughly with soap before washing and disinfecting the device.
- Inspect the parts for any residue after dishwashing. Repeat the washing steps if necessary.

NOTICE

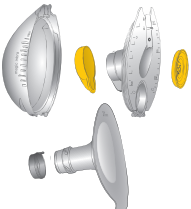
Take care not to damage parts of the breast pump during cleaning:

- Do not immerse the motor unit in water, pour or spray liquid directly on the motor unit.
- Distilled water is recommended when boiling parts to prevent substantial mineral build-up over time, which may compromise your parts.
- If you notice a white residue on the parts after boiling, your water may have a high mineral content.

Information

- By using the dishwasher, parts may become discolored. This will not impact the functioning of the parts.

7.1 Cleaning overview

Motor Units	
	<p>The motor unit is not waterproof. Do not immerse in water. Wipe before first use and as needed by following the steps below:</p> <ol style="list-style-type: none"> 1. Unplug the breast pump from the power outlet and close the cap of the charging port. Confirm that the breast pump is switched off before cleaning. 2. Wipe with a clean towel dampened with warm soapy water. 3. Wipe with a clean towel dampened with cold water. 4. Wipe dry with a clean, dry towel.
Accessory set	
	<p>Always disassemble, pre-rinse and wash all parts before first use and after each use.</p> <p>Disinfect the washed parts before first use and once per day after use.</p> <p style="text-align: right;">  </p>

i Information

Cleaning the Transport Case

- Wipe the outside of the transport case with a towel dampened in soapy water whenever needed.
- Do not remove the foam inserts. If you notice any stains on the foam, do not wash them. Protect each motor unit in a separate clean food-safe plastic bag before placing them into the transport case.

7.2 Washing and disinfecting your Accessory set

Washing and disinfecting are two different activities. They must be done separately to protect you and your baby, and to maintain the performance of your breast pump.

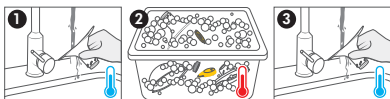
Wash (before first use and after each use)

– to clean the parts by physically removing milk residues and other possible contaminants.


Disinfect (before first use and once per day after use)

– to effectively lower the number of germs that may be present on the surfaces of the washed parts.

- 1. Rinse the disassembled parts, except for the motor unit,** with cold water (approx. 20 °C).
- 2. Wash** the parts thoroughly with plenty of warm, soapy water (approx. 30 °C). Use a mild dish soap.
- 3. Rinse** the parts with cold water for 10 to 15 seconds (approx. 20 °C).

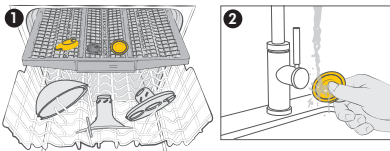OR

- 1. Wash** in dishwasher. Place on the top rack or in the cutlery section in an orientation that prevents the parts from collecting water. Use a commercial-available detergent.
- 2. Rinse the soft silicone parts** (the yellow membrane, the grey shield valve and the yellow valve) under running water after dishwashing.

Boil the washed parts for at least 5 minutes. Make sure the parts are fully immersed in water and that they are not in direct contact with the bottom of the pot.

OR

Disinfect in microwave using Quick Clean™ bags in accordance with the instructions on the bags.

* Refer to local website/shops for availability in your country.

Dry – allow the washed and disinfected parts to completely dry before using or storing them.

Allow the parts to air-dry on a clean and dry dish towel or paper towel. It is important that all residual moisture dries completely.

8. Assembling your breast pump

CAUTION

To keep the pump parts clean and hygienic:

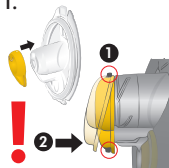
- Wash your hands thoroughly with soap and water before touching any part of the Magic InBra™ Rental breast pump or breasts.
- Dry your hands with a clean hand towel or a single-use paper towel.
- Avoid touching the inside of the milk containers and lids.

To avoid health risks and reduce the risk of injury:

- Before each use, visually inspect the individual components for cracks, chips, tears, deterioration and cleanliness. In the event of damage, refrain from using the Magic InBra™ Rental breast pump until the defective part(s) has been replaced.
- Only use clean and dry parts for assembly. Refer to **chapter 7** for cleaning.


1.

Assemble the yellow valve securely on the container lid.

1. Engage the top hook.
- 2. Push firmly at the bottom to engage the bottom hook.**

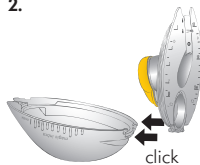2.


Engage the hinge of the container lid to the container. The yellow valve faces the inside of the container.

3.

Close the container lid by pushing it firmly against the container.

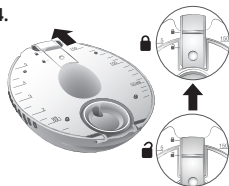4.


Slide the spout lid along the groove to the closed position.

5.


Push the yellow membrane against the container lid to fit it evenly in the groove.

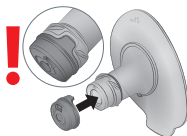6.


Ensuring that the flap is in the correct position, place the grey shield valve on the breast shield.

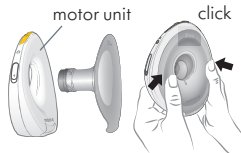7.

Insert the breast shield into the **motor unit** and press it in on both sides until it clicks into place.

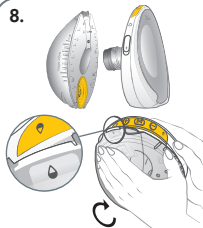8.

Place the **closed** container against the motor unit so that the two drop icons are aligned. Rotate the container clockwise to attach it.

9. Expressing your breast milk

CAUTION

To avoid health risks and reduce the risk of injury:

- Milk expression should not be uncomfortable. Very powerful vacuum, misaligned nipple or incorrect breast shield size can cause trauma to the nipple/breast and decrease milk output. Adjust the breast shield size (see **chapter 15**) and vacuum as needed.
- **Stop using the breast pump if discomfort persists or in the event of negligible milk output. Please contact your healthcare professional or breastfeeding specialist.**
- If you experience pain during pumping, break the seal between the breast and the breast shield with your finger and remove the breast pump from your breast.

Information

- **Stimulation phase** is a fast pumping mode to stimulate milk flow.
- **Expression phase** is a slower pumping mode for gentle and efficient milk removal.
- **Maximum Comfort Vacuum (MCV)** is the highest vacuum setting where pumping still feels comfortable. This is different for every mother and will result in the most efficient milk expression.
- When in Expression phase, increase the vacuum with the + button until pumping feels slightly uncomfortable (not painful). Then decrease the vacuum with one press of the – button. This is your MCV level.
- It is important to identify the MCV during the first pumping session as this vacuum level will be remembered by the breast pump. The breast pump will adjust the vacuum automatically to the MCV level during the next session. While the breast pump adjusts to the MCV level, the + button will be flashing.
- Consider reassessing your MCV each week to ensure you are pumping at your optimal vacuum level.

9.1 Operating your breast pump

The Magic InBra™ Rental breast pump may be worn with your regular nursing bra. Ensure that the bra you are wearing supports the breast pump securely in place while avoiding excessive compression on your breast. Try loosening the bra strap or switching to a more stretchy bra if you experience discomfort.

1.

Ensure to turn on the breast pump before inserting the pump in your bra.

To turn on your breast pump, press and hold the power button until the lights switch on.

2.

The breast pump can be used on the left breast or the right breast. Press the Left selection button or the Right selection button to select which breast you are going to pump.

LR Flashing white; pumping side to be selected

LR Solid white; side has been selected

3.

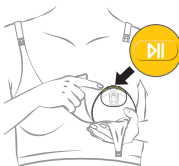
Position the pump on your breast. Ensure that your nipple is centered in the breast shield tunnel.

Tunnel light is on to help you align the nipple.

4.

Hold the pump firmly against your breast and press the Start/Pause button. The pump begins in the Stimulation phase. The + button will be flashing while the vacuum is ramping up.

Tunnel light will switch off after 10 sec of pumping.

5.

Secure the pump in place by hooking nursing bra flap over the pump or by adjusting the bra strap.

6.

Adjust the vacuum with the + button or the - button. There are 5 levels available in the **Stimulation phase**.

7.

Solid white:
Milk detected

When the milk is detected, the pump will switch from the Stimulation to the Expression phase. The + button will be flashing while the vacuum is ramping up. A small noise will be heard when the system fills with milk. This is normal.

8.

Adjust the vacuum with the + button and the - button to your **Maximum Comfort Vacuum** level. There are 10 levels available in the **Expression phase**.

i Information

- In both Stimulation and Expression phase, the + button and the - button are lit up only when it is possible to adjust the vacuum higher or lower. When the lowest or the highest vacuum level is already reached, the breast pump indicates it with one short vibration.
- Please note, for your comfort, the highest vacuum levels in the Expression phase are available only when there is milk expressed.
- The breast pump will switch automatically to the expression phase as soon as milk is detected. If no milk is detected, the breast pump will switch to the Expression phase after 2 min of pumping in the Stimulation phase. If after 15 min of pumping no milk has been detected the pump will pause automatically.
- Magic InBra™ Rental should not be used while lying down, or engaging in strenuous exercise. The pump will notify with two short vibrations and milk status indicator flashing yellow when there is a risk of leakage.

9.2 Realigning the nipple or switching the side

If you need to realign the nipple or switch the side when milk has not yet been expressed:

1. Pause the breast pump by pressing the Start/Pause button.
2. Break the latch with your fingers.
3. Switch the side (if needed) and press the Left selection button or the Right selection button accordingly.
4. Realign the breast pump on your breast.
5. Resume pumping by pressing the Start/Pause button again.

If you need to realign the nipple or switch the side while there is already **milk expressed** follow the steps in **section 9.3** to confirm all the milk is collected into the container. Empty the milk from the container before switching the side and starting a new pumping session.

9.3 Finishing your pumping session

i Information

The maximum volume of the container is 150ml. To avoid the container overflowing, it is recommended to finish your pumping session when the milk fill level reaches the max line on the container (130ml). Please note that up to 20ml will be in the breast shield tunnel until collected into the container during step 3 and 4.

1.

For finishing, first press the Start/Pause button . A slight vacuum remains to prevent leakage.

2.

Unclip your bra supporting the pump against your breast. Lean forward and break the latch with your fingers to gently release your nipple from the pump.

3.

Hold the pump as shown in the image while removing it from your breast. **There will be milk in the breast shield tunnel.**

4.

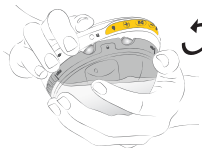
Place the pump on a flat surface in a **horizontal position** for a few seconds. The tunnel light will flash and the remaining milk from the breast shield tunnel will drain into the container. The pump will **switch off automatically**.

10. Pouring milk from the container

i Information


Do not use the container for milk storage or transportation as it is not designed for this purpose.

1.

Once the pump is switched off and no lights are on, rotate the container anti-clockwise to detach it.

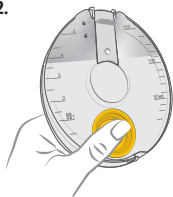2.

You can use the volume scale markings on the container lid to read the estimated collected milk volume while holding the container upright.

3.

click

Slide the spout lid down to open position.

Ensure to pour the milk from the container into a bottle or breast milk storage bag **within few minutes after detaching the container.**

4.

Tilt the container and pour the milk through the pouring spout. After pouring milk from container, disassemble according to the instructions in **chapter 11** and clean parts as described in **chapter 7**.

General breast milk handling guidelines

- Write the date on the storage container. Include your child's name if you are giving the milk to a childcare provider.
- Gently swirl the container to mix the cream part of the breast milk that may rise to the top back into the rest of the milk. Do not shake the milk. This can make some of the milk's valuable parts break down.
- Refrigerate or chill milk right after it is pumped, if possible. You can put it in the refrigerator, place it in a cooler or insulated cooler pack, or freeze it in small (60 to 120ml) batches for later feedings. Pumped milk, at room temperature ($\leq 25^{\circ}\text{C}$) is safe to use without being refrigerated for up to 4 hours after pumping.

Human milk storage guidelines*

Type of breast milk	Storage locations and temperatures		
	Countertop 25°C or colder (room temperature)	Refrigerator 4°C	Freezer -18°C or colder
Freshly expressed or pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, previously frozen	1–2 Hours	Up to 1 Day (24 hours)	Never refreeze human milk after it has been thawed
Leftover from a feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

*Recommended storage times are important to follow for best quality.

For information on storing your breast milk, visit <https://www.medela.com/en-gb/breastfeeding-pumping/articles/pumping-tips/how-to-store-freeze-and-thaw-breast-milk>.

11. Disassembling your breast pump after pumping

1.

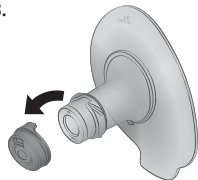
Rotate the container anti-clockwise to detach it.

2.

Detach the breast shield from the motor unit by grasping the tab at the bottom. This may require some force.

3.


Remove the grey shield valve from the breast shield.

4.


Remove the yellow membrane from the container lid.

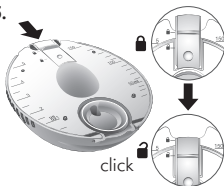5.


Slide the spout lid down to open position.

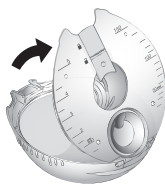6.

Lift the container lid open. This may require some force due to the sealing feature.

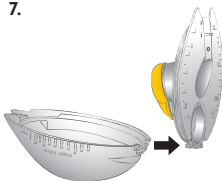7.

Detach the container lid from the hinge.

8.

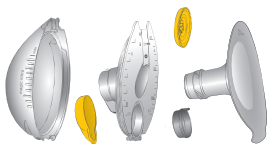
Remove the yellow valve from the container lid.

12. Maintenance and care

12.1 Long-term storage

When not in use for an extended amount of time, the breast pump will automatically transition into storage mode. To remove from storage mode, plug your breast pump into an external power source and charge it.

Before storing your breast pump for an extended amount of time remember to:

Clean the containers, breast shields, membranes and valves (refer to **chapter 7**)

Clean the motor unit (refer to **section 7.1**).

Make sure that the battery status indicator is green.

13. Returning your rental breast pump

At the end of your rental period, please ensure the breast pump is returned in accordance with the terms outlined in your rental agreement with your rental service provider.

1.

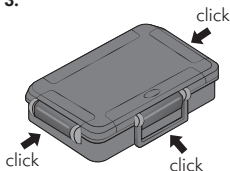
Inspect for Damage. Check the motor units and power cables for any visible damage. If any parts are missing or broken, please notify your rental service provider immediately.

2.

Pack the Pump Securely. Place 2 motor units and 2 power cables back into the original transport case. Ensure all components are included.

3.

Close the transport case carefully. Ensure the three latches are properly closed.

4.

Follow Rental Provider Instructions.

Refer to your rental contract or contact your rental service provider for specific return instructions, including:

- Return shipping method or drop-off location
- Required documentation
- Deadlines for return

i Information

- Do not charge the pump prior to returning it to your rental service provider.
- Do not return the Accessory set. See **chapter 18** for disposal instructions.
- Failure to return the pump in the transport case or by the agreed deadline may result in additional charges.
- Retain proof of return (e.g., shipping receipt or confirmation) for your records.

For questions or assistance with the return process, please contact your rental service provider directly.

14. Troubleshooting

14.1 Vibrations and Notifications

One short vibration: When adjusting the vacuum by pressing the + button or the – button Max/Min Vacuum has been reached.

Two short vibrations: The breast pump will pause automatically after 30 minutes. And also, if the pumping session has not been restarted within 8 min of pausing.

 **Two short vibrations and milk status indicator flashing yellow:** Check pumping position. Risk of leaks. Move to upright position.

 **Two short vibrations and battery status indicator flashing red:** Minimum charge level reached. The breast pump will pause automatically. Finish the pumping session and charge the breast pump afterwards.

 Flashing blue; ready to be paired with the app

 Solid blue; breast pump is connected to the app

14.2 Problems and Solutions

In case of an unexpected behavior of your breast pump check with the troubleshooting table below. Find the issue in bold text, then follow the instructions listed below.

Problem	Solution
<p>Power button not working</p>	<p><u>The breast pump does not switch on:</u></p> <p>Press the power button for 1 second until the lights turn on.</p> <ul style="list-style-type: none"> • Ensure that the breast pump is not on the breast before switching on. • Ensure that the breast pump is charged and the charging cable is not plugged to a power source. • Ensure that the breast pump is not overheated. Allow the breast pump to cool down before switching it on. <p>If the breast pump still doesn't turn on, proceed with a factory reset by pressing and holding the Power and the Start/Pause button for 10 seconds.</p> <p>NOTE: a factory reset will erase the breast pump memory but all data stored on the app will remain accessible.</p> <p><u>The breast pump does not switch off:</u></p> <p>The breast pump cannot be switched off during the pumping session. Press Start/pause button to pause the session, break the latch with your fingers and place the breast pump in a horizontal position until the breast pump turns off automatically (could take up to 5 mins).</p>
<p>Start/pause button not working</p>	<ul style="list-style-type: none"> • Ensure that the breast pump is sufficiently charged (battery status indicator green or yellow). • If the L and R indicators are flashing, select the side.

Problem

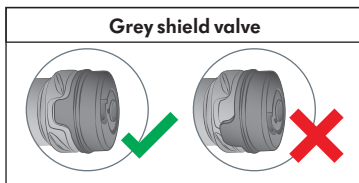
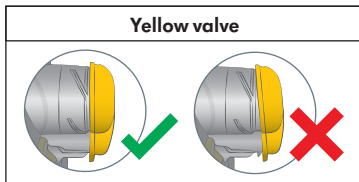
Increase Vacuum (+) button not working

Solution

- The highest vacuum levels can only be reached when milk has completely filled up the breast shield tunnel.
- Ensure that the breast shield is completely sealed against the breast and that there is nothing in between the breast and the shield (for eg. clothes, bra etc.)
- Ensure to boil all parts except the motor unit before first use for proper sealing.
- If the solutions above do not work, disassemble the breast pump following the instructions in **chapter 11**. Ensure all parts are clean, dry and not damaged.
- Reassemble the breast pump correctly, following instructions in **chapter 8**.

Vacuum does not feel strong enough even at high vacuum levels

- The highest vacuum levels cannot be reached until the milk has filled up the breast shield tunnel completely.
- It is possible that the membranes and valves are not assembled correctly. Ensure to assemble all parts according to **chapter 8**.
 - Ensure to assemble all parts according to **chapter 9**.

Problem	Solution
L or R side selection button not working	<ul style="list-style-type: none">• Switching the side is not possible once milk has been expressed. Refer to chapter 9.2.
Pumping is painful or uncomfortable	<ul style="list-style-type: none">• If you experience pain during pumping, break the seal between the breast and the breast shield with your finger and remove the breast pump from your breast.• Ensure that your optimal breast shield size has been selected for maximum efficiency and comfort [see chapter 15].• Ensure you are using your maximum comfort vacuum while pumping [see chapter 9].• Ensure the nipple is centered in the breast shield tunnel.• Ensure that the bra you are wearing supports the breast pump securely in place while avoiding excessive compression on your breast. Try loosening the bra strap or switching to a more stretchy bra if you experience discomfort.
The breast pump pauses unexpectedly	<ul style="list-style-type: none">• Ensure that the breast pump is sufficiently charged (battery status indicator green or yellow).• Ensure that the breast shield is completely sealed against the breast and that there is nothing in between the breast and the shield (eg. clothes, bra etc.)• Ensure to boil all parts except for the motor unit before first use for proper sealing.• If after 15 min of pumping no milk has been detected the pump will pause automatically. Break the latch with your fingers and remove the pump from your breast.• If the solutions above do not work, disassemble the breast pump. Ensure all parts are clean, dry and not damaged. Reassemble the breast pump correctly, following instructions in chapter 8.

Problem	Solution
Battery is not charging	<ul style="list-style-type: none">• Ensure the charging cable is plugged in correctly.• Check if the breast pump is overheated. If yes, let it cool down and try to charge again.• Check the charging port for damage. If damaged, do not charge. Please contact Medela Customer Service.• If the breast pump was not used for a long period of time, it will take longer than usual for the breast pump to start charging (battery status indicator flashing green).
Unable to detach the container from the motor unit	<ul style="list-style-type: none">• Switch the breast pump on and off and try again.
Milk still in the tunnel after finishing pumping session	<ul style="list-style-type: none">• To collect milk from the tunnel into the container, follow the instructions in chapter 9.3.• If a little milk is still left in the tunnel, detach the container from the motor unit holding it in horizontal position. Open the spout lid and pour the milk from the container into the bottle. The remaining milk in the breast shield tunnel will be directed into the bottle while pouring.
The motor unit exterior got wet with / or has been submerged in any liquid	<ul style="list-style-type: none">• If a device has been exposed to water or other liquids, do not touch, unplug the device from electrical outlet, turn off and contact manufacturer.• Contact Medela Customer Service.
The breast pump exterior got wet	<ul style="list-style-type: none">• If plugged, unplug the breast pump from the power source and turn it off.• Dry off the exterior of the breast pump.

Problem	Solution
Breast pump lights flashing unexpectedly (3 or more flashing at the same time)	<ul style="list-style-type: none">• The Magic InBra™ Rental breast pump regularly checks its system to ensure everything is working correctly and to detect any unauthorized changes or security breaches. In case of a failure, the pump generates an error and may stop working.• Turn the pump off and proceed with a factory reset by pressing and holding the Power and the Start/Pause button for 10 seconds.• If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service for assistance.
The breast pump battery gets depleted prematurely or unexpected changes in settings occur (e.g. vacuum level changes)	<ul style="list-style-type: none">• Turn the pump off and proceed with a factory reset by pressing and holding the Power and the Start/Pause button for 10 seconds.• If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service for assistance.

If the device is not functioning as expected, turn the pump off and proceed with factory reset by pressing and holding the Power and the Start/Pause button for 10 seconds.

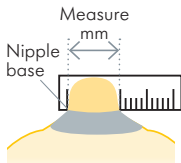
If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service. For contact data visit www.medela.com. Under "Country" choose your country.

15. Checking if you have the correct breast shield size

⚠ CAUTION

To avoid health risks and reduce the risk of injury:

- It is important to verify your optimal breast shield size for maximum comfort and pumping efficiency. Incorrect breast shield size can cause trauma to the nipple/breast and decrease milk output.
- Using a ruler or measuring tape measure the diameter of your nipple at base in millimeters (mm).
 - Based on your measurement, select the breast shield that is slightly larger than your nipple.
 - Test your breast shield size to ensure pumping feels comfortable.

Your nipple diameter (mm)			
14 mm or smaller	15 mm, 16 mm or 17 mm	18 mm, 19 mm or 20 mm	21 mm, 22 mm or 23 mm

*For ordering information about accessories see **chapter 16**.

You can use the lines on the breast shield to help you find the right size as described in the images below.

If you are unsure about correct breast shield sizing please visit www.medela.com/fittingguide or see a lactation consultant/breastfeeding specialist who can help you find the proper fit.

16. Accessories available for purchase

In case of difficulties in finding your desired accessory, please contact Medela Customer Service. For contact data visit www.medela.com and choose your country. For replacement of lost or defective parts please contact Medela Customer Service. Other Medela products are available at www.medela.com.

Accessories

Article number	Product
Depending on your region, check local website/shops for availability.	Magic InBra™ Breast shields 18mm*
	Magic InBra™ Breast shields 21mm*
	Magic InBra™ Breast shields 24mm*
	Magic InBra™ Breast shields 27mm*
	Magic InBra™ Containers
	Magic InBra™ Membranes
	Magic InBra™ Valves
	Magic InBra™ Shield Valves

* If the result of expression is unsatisfactory or expression is painful, contact your lactation consultant or breastfeeding specialist. A different size of breast shield may help make expression more comfortable and effective.

17. Warranty

Find information on the international warranty on www.medela.com/ewarranty.

18. Disposal

Breast shields, container, container lid, membranes and valves are made of plastics and may follow separate collection according to local collection and recycling systems.

In the European Union, United Kingdom and Switzerland the manufacturer or its vendor must take back EEE waste equipment, other countries may have similar collection and recycling systems. This appliance contains batteries that are non-replaceable and must not be removed for disposal.

NOTICE

Dispose of waste equipment according to local regulations. Inquire at the point of sale or contact your local authority for appropriate collection points for waste equipment. The separate collection and recycling of your waste equipment at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

19. Meaning of symbols

The following tables explain the meaning of the symbols found on the product parts and its packaging.

Indicates the location of the Power button

Indicates Start/Pause button

Keep away from rain.
Keep in dry conditions.

Indicate increase/decrease vacuum buttons

Indicates milk status

General safety alert symbol, points to information related to safety

Indicates battery status

Indicates the selection of left/right breast for pumping

IP22

Indicates the degree of protection against ingress of foreign objects and moisture.

Indicates that this device contains Bluetooth wireless technology. (Trademarks of Bluetooth Special Interest Group (SIG))

Indicates the manufacturer

Indicates compliance with international requirements for protection from electric shock (Type BF applied parts).

EU REP

Indicates the authorized representative in the European Union

Indicates the importer

Indicates that the package is capable of being recycled

Indicates the date of manufacture

REF

Indicates the catalogue number

Indicates that the contents of the transport package are fragile and the package shall be handled with care.

Indicates the serial number

Indicates the batch code / lot number

The packaging contains products intended to come in contact with food according to Regulation (EC) No 1935/2004

Indicates the item is a medical device

Read and follow the instructions for use.

Defines a relative humidity range

Defines a temperature range

Do not dispose of electric/ electronic devices together with unsorted municipal waste (dispose of the device in accordance with local regulations).

Defines an atmospheric pressure range

Keep away from sunlight

Indicates the material from which an item is made

Indicates a carrier that contains unique device identifier information.

Indicates direct current.

The Regulatory Compliance Mark indicates the compliance with Australian EMC and safety requirements.

Indicates the model number of the device

Country specific

Indicates compliance with the EU requirements for medical devices

Indicates that an item can technically be recycled according to the French AGEC law.

35

Indicates that the product in the package meets the requirements of Standard GB4806 and is intended to be in direct contact with food.

Indicates that an item can be technically recycled according to the Royal Decree 1055/2022 (Spain)

20. International regulations

20.1 Electromagnetic compatibility (EMC)

The Magic InBra™ Rental breast pump complies with all requirements of the relevant standards and regulations with regard to electromagnetic emission and immunity to interference.

The essential performance functions include vacuum not exceeding 350 mmHg and backflow protection to prevent milk leakage into the motor unit. The Magic InBra™ breast pump was tested for immunity to electromagnetic disturbances and passed using the criteria of maximum vacuum, backflow protection and operation of the breast pump.

WARNING

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Portable wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies, RFID, other electrical equipment and also including peripherals such as antenna cables and external antennas can affect the electric breast pump and should be kept at a distance of at least 30 cm away from the breast pump including cables. Otherwise, degradation of the performance of this equipment could result.
- Use of USB cables other than those provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Guidance and manufacturer's declaration – electromagnetic emissions

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.

Emission tests	Compliance	Electromagnetic environment – guidance
RF Emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.

The Magic InBra™ breast pump was tested for immunity to electromagnetic disturbances and passed using the following criteria:

1. No change in vacuum level and operation of the breast pump.
2. The pump can be controlled.
3. The Bluetooth® communication may be impaired temporarily but may never be completely interrupted in such a way that a new pairing is necessary.

Reset (The following sequence can be used to reset the device in the event of an EM Disturbance)

1. Ensure the breast pump is not attached to the breast.
2. Ensure the device is powered off.
3. Disconnect the power cable.
4. Disconnect the power adapter from the wall outlet.
5. Press and hold the Power and the Start/Pause button for 10 seconds.
6. Wait 10 seconds.
7. Press the Power button for 1 second until the lights turn on to power on the device.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 8 kV contact discharge +/- 15 kV air discharge	+/- 8 kV contact discharge +/- 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 5%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0,5 kV, ± 1 kV Line-to-line	± 0,5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0 % UT for 1 cycle at 0° 70 % UT for 25 cycles at 50 Hz at 0° for 30 cycles at 60 Hz at 0° 0 % UT for 250 cycles at 50 Hz at 0° for 300 cycles at 60 Hz at 0°	0 % UT for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0 % UT for 1 cycle at 0° 70 % UT for 25 cycles at 50 Hz at 0° for 30 cycles at 60 Hz at 0° 0 % UT for 250 cycles at 50 Hz at 0° for 300 cycles at 60 Hz at 0°	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump requires continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	It may be necessary to position the breast pump further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz outside ISM band	3 Vrms	Except as indicated in the table on the next page, portable and mobile RF communications equipment should be used no closer to any part of the breast pump, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommend separation distance $d = 1.2\sqrt{P}$
	6 Vrms 150 kHz to 80 MHz in ISM and amateur radio bands	6 Vrms	$d = 2.0\sqrt{P}$
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	$d = 0.35\sqrt{P}$ 80 MHz to 800 MHz $d = 0.7\sqrt{P}$ 800 MHz to 2.7 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, (a) should be less than the compliance level in each frequency range (b). Interference may occur in the vicinity of equipment marked with the following symbol.
	3 V/m 2.7 GHz to 6 GHz	3 V/m	 A minimum distance of 30 cm is recommended for proximity wireless communication equipment.

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Table of frequencies of portable and mobile transmitters for which the recommended separation distance is 30 cm (12 inches):

Band (MHz)	Service
380 - 390	TETRA 400
430 - 470	GMRS 460, FRS 460
704 - 787	LTE Band 13, 17
800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5
1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS
2400 - 2570	Bluetooth®, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7
5100 - 5800	WLAN 802.11 a/n

20.2 Simplified EU Declaration of Conformity

Hereby, Medela declares that the radio equipment type Magic InBra™ Rental is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address:

www.medela.com/eu-conformity-magic-inbra-rental

21. Technical specifications

Summary of important technical specifications

Vacuum performance:

-75 mmHg to -300 mmHg

45 to 70 cpm

Transmission standard: Bluetooth® Low Energy 4.2


Bluetooth range: up to 10 m

Bluetooth encryption: 128-bit AES-CCM for secure connections


Frequency range: 2400-2483.5 MHz

Max. power output: +4 dBm


Battery capacity and type	3.7V, 970 mAh (rated) Li-ion
Battery runtime	Approximately 2.5 hours
Size	126 x 112 x 71 mm
Weight	200 g ±10 g
Ingress protection level	IP22

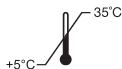
Operating temperature range
5°C to 35°C

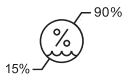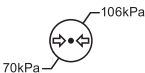
Operating humidity range
(15% to 90% relative humidity)

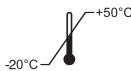Operating ambient pressure range
(70 kPa to 106 kPa)

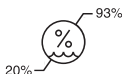Transport/storage temperature range
-20°C to 50°C

Transport/storage humidity range
(20% to 93% relative humidity)

Materials touching skin or coming in contact with milk

- Breast shield: Polypropylene
- Container: Polypropylene
- Container lid: Polypropylene, thermoplastic elastomer
- Yellow membrane, grey shield valve and yellow valve: silicone
- Motor unit: PC/ABS (polycarbonate/acrylonitrile butadiene styrene)
- Yellow buttons area: silicone

All parts that come in contact with breast milk are not intentionally made of BPA (Bisphenol A).

Recommended power adaptor specifications

USB-A Power supply (USB-A to USB-C cable provided)

5V / 1A DC

Compliant with IEC 62368-1 standard and corresponding to the regional power source regulations.

Battery Information

Battery Type: Rechargeable Li-Ion Battery

Manufacturer: Chongqing VDL Electronics Co., LTD.

Building 1-4, Puli Industrial New Area, Zhaojia Street,
Kaizhou District, Chongqing, 405401 China

Email: vdl@gdvd.com

Capacity: Rated capacity 970mA, Nominal voltage 3.7V, Rated Energy 3.59Wh

Weight: 17g

Made in China

22. Cybersecurity information

This section provides essential information about the security features of the Magic InBra™ Rental breast pump.

i Information

- The charging port on the breast pump is intended for charging purposes only.
- Do not connect the breast pump to any device other than the recommended power adaptor (see **Chapter 21** for technical specifications).
- Only use the supplied USB power cable to charge your breast pump.

Scan for the Security Information website.

Note: The information provided via the QR code supplements but does not replace the Instructions for Use.

Factory reset

You can perform a factory reset by pressing and holding the Power and Start/Pause buttons for 10 seconds. The factory reset allows you to restore your Magic InBra™ Rental breast pump to its original factory settings if the device is not functioning as expected and/or other troubleshooting steps have failed.

Note: A factory reset will erase all breast pump memory. If the app is being used, all data stored in the app will remain accessible. For further information on data stored in the app, please refer to the Instructions for Use of the Medela Family Pump Control app.

Software Bill of Materials (SBOM)

To ensure transparency and security, Medela provides upon request a SBOM detailing the components used in the device. Please contact Medela Customer Service for assistance.

Secure firmware updates and patches

The breast pump supports secure firmware updates to address potential security vulnerabilities. Firmware updates are provided through Medela Customer Service. To maintain the device security, it is recommended to check for routine security updates and patches at the following website regularly:

<https://www.medela.com/en/footer-content/magic-inbra-security-information>.

End of support

Medela will provide device support, including secure firmware updates during the device lifetime. Users can find detailed information about this policy on our website at

<https://www.medela.com/en/footer-content/magic-inbra-security-information>.

Mobile app compatibility

To ensure optimal performance and security, it is essential to use the mobile app version compatible with your device's operating system. You can check your Medela Family Pump Control app's compatibility by visiting the following website:

<https://www.medela.com/en/footer-content/magic-inbra-security-information>.

Ensure that your Medela Family Pump Control app is updated to the latest version to benefit from security patches and feature enhancements.

Bluetooth security

The Magic InBra™ breast pump utilizes Bluetooth Low Energy to securely transmit data, enabling remote control of the breast pump's operations and sharing of pump session information through the Medela Family Pump Control app. The Bluetooth Low Energy connection is secured with industry-standard encryption to prevent unauthorized access and data interception, ensuring user privacy and data security, and that the pump can only be controlled within a limited distance. In case the app is being used, and the breast pump has an unstable connection with the app, please refer to the Instructions for Use of the Medela Family Pump Control app to further assistance.

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© Medela AG ref. master doc. 101048517/A, 101048231/2025-10/A
GmbH & Co. Handels KG
Medela UK Ltd.