Contact Nipple Shield Sterile

Comfortable temporary help for breastfeeding mums.

Please read all instructions before using this product.

Intended use

The Medela Contact Nipple Shields are intended to cover the nipples of a mother during breastfeeding.

Product Description

A Contact™ Nipple Shield is a temporary solution which should be appropriately sized and fitted to the mother's nipple and used under professional guidance.

Where to find & purchase it:


You can find the Contact™ Nipple Shields in pharmacies if you wish to buy some more. Our retail version consist of two Contact™ Nipple Shields with a storing box and can be reused multiple times. Please follow the instruction for use for cleaning and handling.


Contraindications

There are no known contraindications.


How to apply your Contact™ Nipple Shields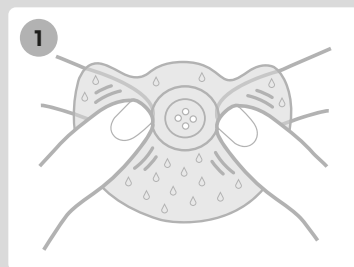


1. Wash your hands thoroughly with drinking-quality water and soap before touching the shield and the breast. Dry your hands with a fresh towel or a single-use paper towel. Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples. Moisten the shield with drinking quality water as this will help it to stick to the skin (see figure 1).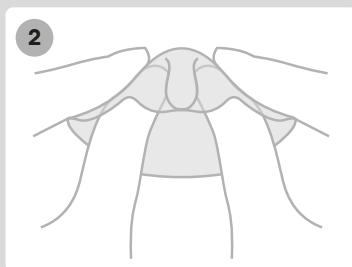

2. Using your thumbs, push the tip of the shield halfway in (see figure 2).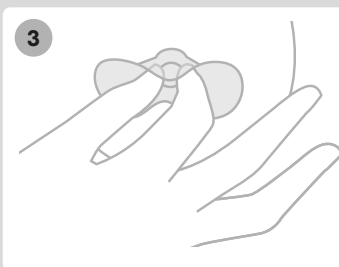

3. Place the shield centrally over the nipple so the cut-out will face your baby's nose during feeding (see figure 3).

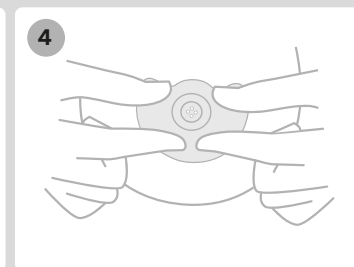4. Push down on the wings until the shield pops out again and a seal is formed. (see figure 4).

Important safety information

- Use the product only for its intended use as described in this instruction for use.
- If problems or pain occur, consult your lactation consultant or physician.
- Inspect before each use. Dispose at the first signs of damage or weakness, or if mildew on it.
- Never leave packaging and components unattended. Keep them out of reach of children.
- Contact™ Nipple Shield can be reused during breastfeeding session within 24 hours by rinsing it with clean water.

Choosing the correct nipple shield size

- When choosing the correct nipple shield size you need to consider both mother and baby.
- For the mother: The shield should fit comfortably without pinching.
 - In the case of a premature infant: When grasping the shield, the infant's lips must be able to reach the flat base of the shield against the breast.

Meaning of symbols

-  E. Indicates that the device can be used multiple times only for a single patient (ISO 15223-1, 5.4.12)
-  F. Indicates the manufacturer (ISO 15223-1, 5.1.1)
-  G. Sterilized using ethylene oxide (ISO 15223-1, 5.2.3)
-  H. Indicates the manufacturer's product number (ISO 15223-1, 5.1.6)
-  I. Indicates the UDI number (ISO 15223-1, 5.7.10)
-  J. Open here (Sterile barrier system) (ISO7000:3079)
-  K. Indicates the date of manufacturing (ISO 15223-1, 5.1.3)
-  L. Indicates the manufacturer's batch code or lot (ISO 15223-1, 5.1.5)
-  M. Indicates that the packaging contains products intended to come in contact with food according to regulation (EC) No 1935/2004

Product variants

Available in 3 sizes: S/16mm, M/20mm or L/24mm
Material nipple shield: Silicone
Disposal: According to local regulation

Meaning of symbols

-  A. Keep Dry (ISO 15223-1, 5.3.4)
-  B. Keep away from sunlight (ISO 15223-1, 5.3.2)
-  C. Consult the instructions for use (ISO 15223-1, 5.4.3)
-  D. Do not use if package is damaged (ISO 15223-1, 5.2.8)

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