



**neonatal  
solutions**

## Directions for Use

Dose Cap

ENFit

R<sub>x</sub> Only

Not made with natural rubber latex.

Not made with Diethylhexylphthalate (DEHP).

**NOTE:** ±10% transfer volume requires minimum 0.7 mL delivery.

### WARNING:

- The ENFit Dose Cap is not intended for intravascular or intravenous use.
- This device has the potential to misconnect with small bore connectors of other healthcare applications.

### CAUTIONS:

- Contents are sterile in an unopened, undamaged package. Do not use this device if opened or damaged.
- Do not re-sterilize.

### INDICATIONS FOR USE:

- The ENFit Dose Cap is intended to be attached to an enteral extension set, nasogastric feeding tube and/or enteral syringe. The ENFit Dose Cap is disposable and single use only.

### RECOMMENDED PROCEDURE:

- Use clean technique.
- Ensure this device is only connected to an enteral port and not to an I.V. set.

### REMOVAL:

- Remove the ENFit Dose Cap and discard per hospital protocol.