Medela Family Pump Control

Smartphone app

Instructions for use

ENGLISH

Visual appearance of pump and app might vary from pictures.
The amount of milk pumped using this product might vary between users.
Software version v1.21

Contents

1. Important safety information	3
2. Indications and contraindications	3
2.1 Intended use / purpose	3
2.2 Contraindications	3
3. Product description	3
4. Getting started	4
4.1 System requirements	4
4.2 Supported pumps	4
4.3 Installing and opening the app	4
4.4 App and operating system updates	4
5. Pairing the breast pump(s)	5
6. Starting a pumping session	8
6.1 Selecting pump sides	8
6.2 Nipple alignment	9
6.3 Notifications	10
6.3.1 Pump connection status	10
6.3.2 Milk detected	10
6.3.3 Check pumping position	10
6.3.4 Check container fill level	10
6.3.5 Error detected	11
6.3.6 Low battery level	11
6.3.7 Very low battery level	11
6.3.8 Vacuum level not reachable	11
6.3.9 Pump not responding	11
6.3.10 Pump time count not be updated	11
6.3.11 Pump timer	11
6.4 Pumping Screen	12
6.5 Controlling the pump lights	14
6.6 Timer	15
7. Finishing your pumping session	16
8. Adjusting the volume and saving a pumping session	18
9. Managing and exporting session history	19
10. More	22
11. Unpairing the pump	23
12. App uninstall	23
13. Troubleshooting	24
14. Meaning of symbols	25

1. Important safety information

▲ Please read all instructions before using this product.

The following symbols and signal words show the significance of the information:

▲ CAUTION

Can lead to minor injury.

NOTICE

Can lead to material damage (not related to personal injury).

i Information

Useful or important information that is not related to safety.

▲ CAUTION

To avoid health risks and reduce the risk of injury:

- Milk expression should not be uncomfortable. Very powerful vacuum, misaligned nipple or incorrect breast shield size can cause trauma to the nipple/breast and decrease milk output. Adjust the breast shield size and vacuum as needed.
- Stop using the breast pump if discomfort persists or in the event of negligible milk output. Please contact your healthcare professional or breastfeeding specialist.
- If you experience pain during pumping, break the seal between the breast and the breast shield with your finger and remove the pump from your breast.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices): if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to Medela AG and/or its authorised representative and to your national authority.

2. Indications and contraindications

2.1 Intended use / purpose

The Medela Family Pump Control app enables you to connect your Medela breast pump to your smartphone. Once connected, the app allows you to control your pump remotely. Note, the Medela Family Pump Control app is not required to use your Medela pump. You can control your Medela pump directly.

The Medela Family Pump Control app displays information on the active pumping session and saves the session history in the app. The app allows you to manually manage session history information and to export it for personal use only.

2.2 Contraindications

There are no known contraindications for the Medela Family Pump Control app.

3. Product description

The Medela Family Pump Control app can connect to your Magic InBra™ breast pump via Bluetooth®.

Trademark recognition: The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Medela AG is under license.

4. Getting started

4.1 System requirements

The app can be installed on iPhone and Android smartphones. The app is not supported on tablets.

System requirements:

- iOS version: 18, 26
- Android version: 14, 15, 16
- 100MB memory storage
- Bluetooth (required to exchange data with the Magic InBra breast pump)

We spend commercially reasonable effort to ensure compatibility with most smartphones but can not guarantee that the app will be compatible with every smartphone.

4.2 Supported pumps

Magic InBra breast pump.

4.3 Installing and opening the app

1. Ensure that the internet connection is available to download the app.
2. Open the App Store or Google Play app on your smartphone.
3. Search for Medela Family Pump Control.
4. Tap the download button. You might need to enter your App Store or Google Play email and password to confirm the download.
5. Once installed, tap the Medela Family Pump Control app icon to open the app. After installation, no internet connection is required to use the app.

Android, Google Play and the Google Play logo are trademarks of Google Inc. iPhone®, Apple® and the Apple logo are trademarks of Apple Inc. App Store® is a service mark of Apple Inc.

By downloading the app you agree to the End-User License Agreement of the respective app store.

Security Notice

For secure operation, install the app only on your smartphone. This app is intended for single-user access; do not share your smartphone credentials with others.

4.4 App and operating system updates

Please ensure both your operating system (iOS or Android) and app are always up to date by enabling automatic updates on your smartphone for optimal performance and security. Updates might include security patches.

On iOS:

1. Go to the Settings app.
2. Scroll down and tap App Store.
3. Activate App Updates.

On Android:

1. Open the Google Play Store app.
2. Tap the upper right corner on your user profile picture.
3. On the bottom of the pop-up menu select Settings.
4. Tap Network Preferences and then Auto-update apps.

i Information


To maintain the security and functionality of the app, routine updates and patches are provided through the app stores.

5. Pairing the breast pump(s)

Ensure your Magic InBra breast pump(s) and your smartphone are charged before use. For more information, please refer to the Instructions for Use of the Magic InBra breast pump and your smartphone.

When using the app for the first time, the operating system of your smartphone will ask for permission for the app to use Bluetooth (iOS) or Nearby devices (Android). You need to give permission otherwise you will not be able to pair your breast pump(s).

iOS:

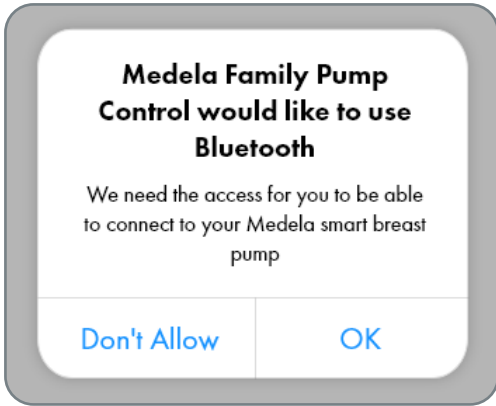Android:

OR

5

Pairing the breast pump(s)

Follow the on-screen instructions for a step-by-step guide on how to securely pair your breast pump(s) via Bluetooth. If you have two breast pumps, pair each pump one after the other. Keep the breast pump within 20 centimeters / 8 inches of your smartphone during the process for successful pairing.

i Information

- The app pairs with only one breast pump at a time to prevent unauthorized connections.
- Avoid pairing in public locations to reduce the risk of unauthorized connections.
- Use the Bluetooth connection only with your personal smartphone.

1. To turn on your breast pump, press and hold the power button until the lights switch on.

Important: If you have two pumps, please turn on only one of your pumps!

- 2. To start pairing press and hold the START/PAUSE button (▶||) for 5-10 seconds until the Bluetooth light on the pump is flashing blue.

Press and hold the START/PAUSE button on the pump until the Bluetooth light on the pump is flashing blue. This takes approximately 5-10 seconds.

CANCEL

- 3. To finalize the pairing, the app will ask you to press either the "-" or "+" button on your breast pump.

Press the "-" button on the pump

or


Press the "+" button on the pump

- Once you have successfully paired your breast pump, tap the “Pair a second pump” to pair your second pump or tap the “Start pumping session” button to start pumping. Next time you turn on the pump, it will be connected automatically.

It is not possible to pair two pumps simultaneously. Please turn off the second pump if you are seeing this error.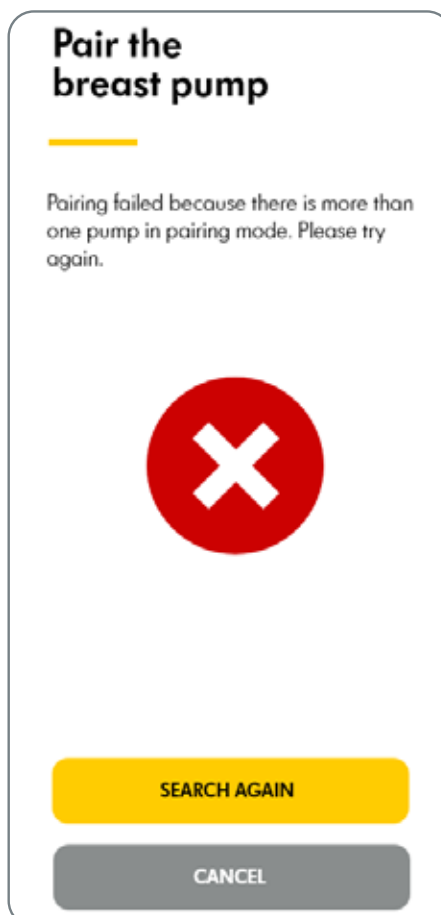

If no pump could be found, please refer to chapter 13 “Troubleshooting” in this document.


6. Starting a pumping session

When using the app for the first time, on-screen help will guide you through the pumping user interface. Tap the screen to move through the screens. To see the on-screen help again, go to MORE and then tap “Quick start screens”. The content of the on-screen help is the same as described in the following subchapters. The pumping screen adjusts dynamically based on whether you have one or two pumps connected.


6.1 Selecting pump sides

Before starting a pumping session, you’ll need to assign a side to each pump. Press the Left or the Right side selection button on the pump. If you have two pumps, the second pump will be automatically assigned with the other side.

If the side has not been assigned, the app will display a pop-up to guide you through the side selection process.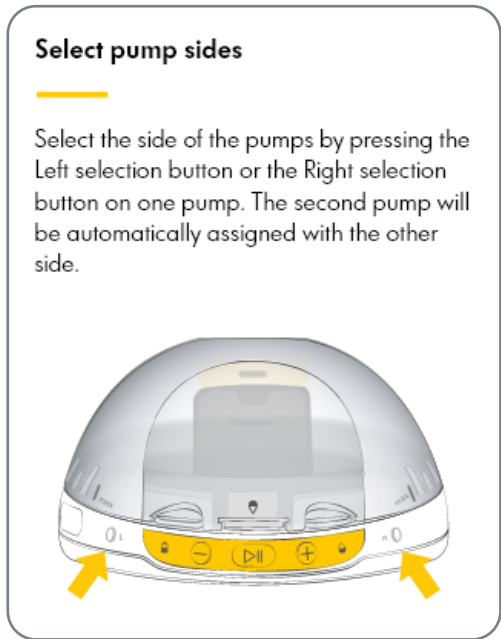

If both pumps are assigned to the same side, the app will display a pop-up with instructions on how to correct that.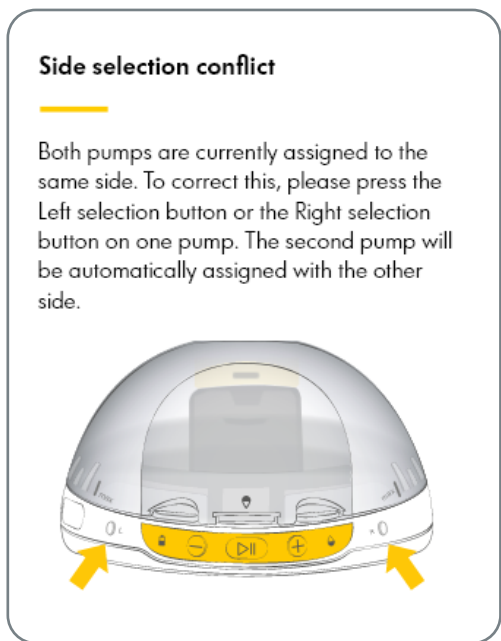

If both pumps are assigned to the same side and the first pump is already pumping, the app will automatically assign the second pump to the other side.

Side selection conflict

The side you have selected on your second pump is already taken by your first pump. The second pump will be automatically assigned with the other side.

If both pumps are assigned to the same side and both pumps are already pumping, the app will prevent remote control of the pump as the app cannot differentiate between left and right. Switching the side while pumping is not possible.

App controls disabled because both pumps are assigned to the same side. Please use the buttons on the pump.

Before starting the next pumping session, ensure to have each pump assigned to a different side.

Note: If you are planning to use only one breast pump on both breasts, please finish the session on the first breast (see chapter 7 “Finishing your pumping session”), empty the container and then start a new pumping session on the second breast.

6.2 Nipple alignment

Ensure your nipple is centered in the breast shield tunnel as shown in the app’s illustration.

Nipple Alignment

Position the pump on your breast. Make sure your nipple is centered in the breast shield tunnel.


Never show this again

OK


6.3 Notifications

When you first launch the app, your smartphone's operating system will prompt you to allow notifications from the app. This notification permission is necessary to keep you informed about your pumping sessions and other app and pump activities.

iOS: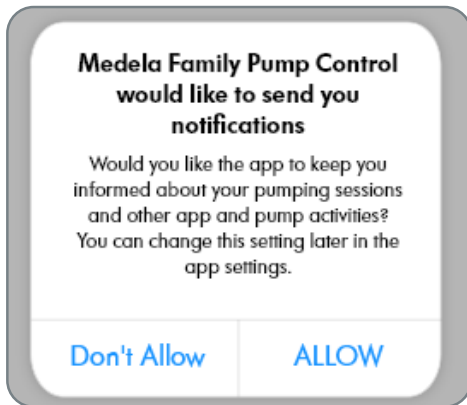

Android:

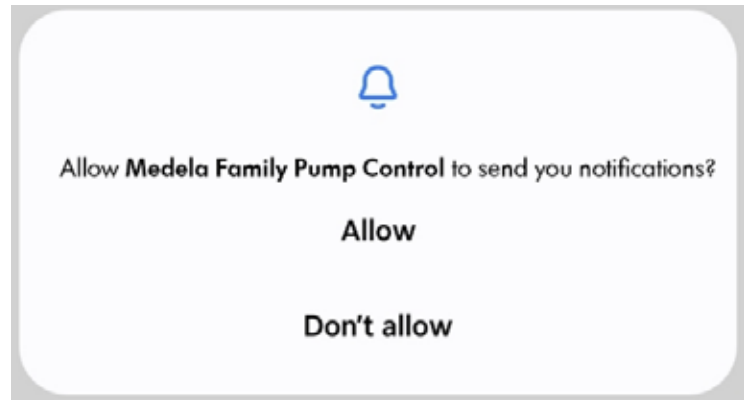
OR

10

At the beginning of every notification the app will show the side of the pump affected.


6.3.1 Pump connection status

The app will show the connection status of the pump.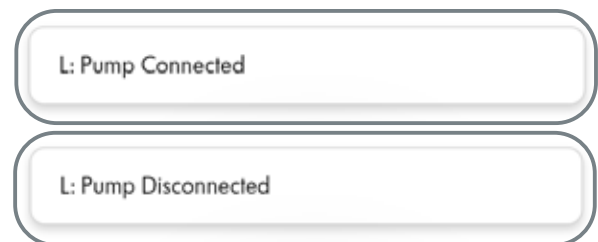


6.3.2 Milk detected

As soon as milk is detected the app will show the following pop-up.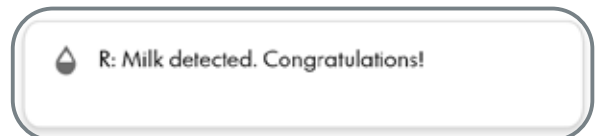


6.3.3 Check pumping position

Risk of leaks. Move to upright position.

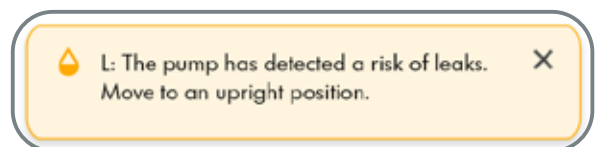
Once the position has been corrected, the app will show the following pop-up.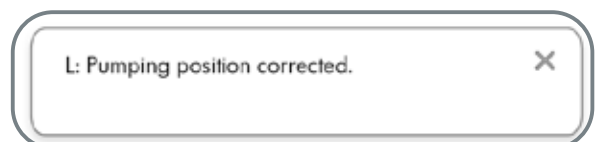

6.3.4 Check container fill level

To avoid overflow:
When the milk fill level reaches the max line proceed to finish the session by following the guidance on chapter 7. There will be milk in the breast shield tunnel. It will be collected automatically into the container during the finishing process.

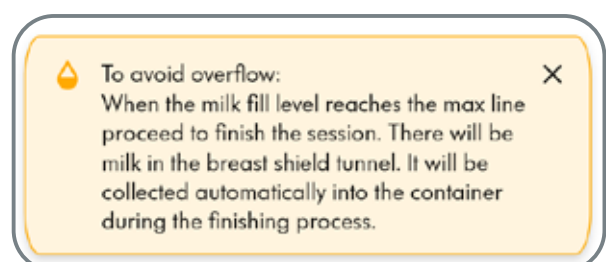6.3.5 Error detected

When no milk has been detected yet:
Pump paused due to an error detected.
Please remove the pump from your breast.
Then turn the pump off, disassemble the pump and ensure to assemble all parts correctly according to the instructions for use. Ensure that the nipple is aligned in the middle of the breastshield tunnel and try pumping again.

When milk has been detected already:
The pump has been detached from the breast.
Lean forward and finish your pumping session by placing the pump in a horizontal position.

6.3.6 Low battery level

When the pump battery level is low, the app will show this pop-up. Please charge your pump before the next pumping session.

6.3.7 Very low battery level

When the pump battery level is very low, the app shows this pop-up. Please charge your pump before the next pumping session.

6.3.8 Vacuum level not reachable

The highest vacuum levels can only be reached when milk has completely filled up the breast shield tunnel. If the selected vacuum level can not be reached, the app will display this pop-up.

6.3.9 Pump not responding

If the pump is not responding to the app control (e.g. Bluetooth disconnection), the app will display this pop-up.

6.3.10 Pump time count not be updated

The app displays this pop-up when a pump is connected and the app can not set the time on the pump. Please turn off the pump and turn it on again to solve this issue.

6.3.11 Pump timer

After setting the timer, this notification will be shown when the time is up.

L: Pump paused due to an error detected. Please check the pump to see if all parts are correctly assembled. Ensure that the nipple is aligned in the middle of the breastshield tunnel and try pumping again. ✕

L: Pump has been detached from the breast. Lean forward and finish your pumping session by placing the pump in a horizontal position. ✕

R: Battery running low. Please charge your pump before your next pumping session. ✕

R: Pumping stopped due to very low battery. Press the Finish button to collect the milk into the container. Please charge your pump before your next pumping session. ✕

L: Vacuum level cannot be reached. For your comfort, higher vacuum levels can only be reached when milk has completely filled up the breast shield tunnel.

L: The pump is not responding

L: The pump time could not be updated. Try turning the pump off and back on.

Pump paused based on the timer set. Continue to finish your session. If you want to continue pumping, press Cancel and then the START/PAUSE button. ✕

6.4 Pumping Screen

Ensure you always turn on the breast pump before inserting the pump in your bra. After successfully pairing the pump, it will automatically connect next time you turn it on.

Single pumping screen

Double pumping screen

Before starting the first pumping session, the app will show a Quick Start guide to get familiarized with the app.

On the top of the screen you will see

- the side selected (L or R)
- the battery status in percentage



At the bottom of the screen you can tap the yellow START button to start pumping. When double pumping, pressing the START button will start both pumps simultaneously.

The START button turns into the FINISH button once you start pumping.

On the bottom right corner of the screen you will find the total duration.

The circular yellow progress bar increases indicating the approximate amount of pumped milk.

It is recommended to still keep an eye on your pump to ensure the milk fill level does not exceed the max line on the container.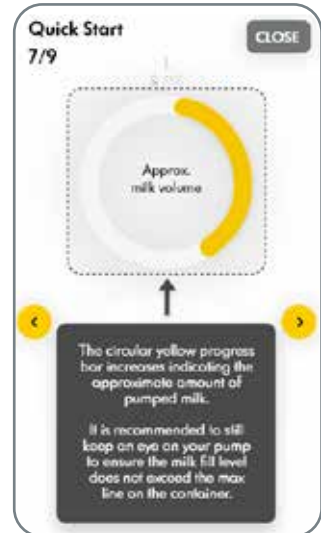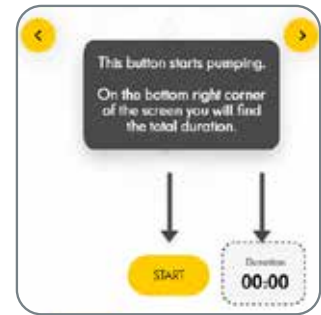


Below the circle you can see the current phase of each pump: Stimulation or Expression.


i Information

- **Stimulation phase** is a fast pumping mode to stimulate your milk flow.
- **Expression phase** is a slower pumping mode for gentle and efficient milk extraction.
- **Maximum Comfort Vacuum (MCV)** is the highest vacuum setting where pumping still feels comfortable. This is different for every mother. To find your MCV increase the vacuum level by tapping the + button until pumping feels slightly uncomfortable (not painful), then decrease the vacuum slightly with one tap of the - button.


The pump starts in Stimulation phase and automatically switches to Expression phase as soon as milk is detected or after 2 minutes if no milk is detected. You can switch the phase by tapping the icons shown to the right.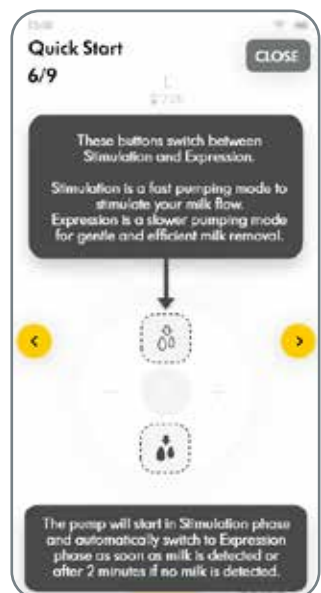

In this screen you can also find the vacuum level controls as well as the current vacuum level. When the vacuum is already adjusted to the maximum level, the + button will be greyed out; conversely, when the vacuum is adjusted to the minimum level, the - button will be greyed out.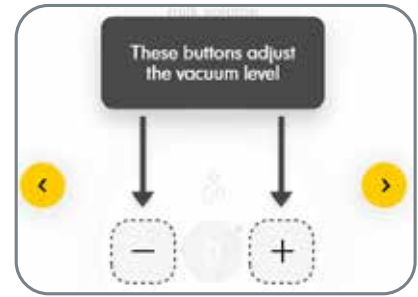

♥ The heart icon next to the vacuum level indicates the last stored Maximum Comfort Vacuum (MCV) level. The pump stores the Maximum Comfort Vacuum (MCV) level, and will automatically adjust to this level during your next pumping session. Please refer to the Instructions for Use of your Magic InBra breast pump for more information.

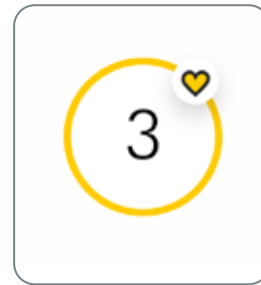You can pause the session anytime by tapping the PAUSE (||) button. To resume pumping tap the START (▶) button.

i Information

- If you move away from your smartphone and/or the Bluetooth connection is lost during a pumping session, the app might lose the connection to your pump(s). The app will notify you of the disconnection but the breast pump will continue to operate.
- When you return close to the smartphone and/or the Bluetooth connection is restored, the app will automatically reconnect.

To pair a new pump, access app settings, or find additional help, tap "MORE" at the bottom of the screen.

6.5 Controlling the pump lights

Tap the lamp icon on the upper right corner to turn the pump interface lights on and off.

Pump lights on

Pump lights off

6.6 Timer

Tap the timer icon on the upper right corner to set a timer.

A pop-up will open to allow you to select the minutes for the timer. Please note that the pump pauses automatically after 30 minutes.

If the Bluetooth connection to the pump is lost during the countdown, the timer will be deactivated.

Tap OK to close the pop-up. On the pumping screen, pressing the START button will start both pumps simultaneously.

To use the timer and get informed when the time is up, you need to give the app permission to send you push notifications. Please give the app permission to continue. (see Chapter 6.3 for more information).


When the timer is running, the app will show the timer countdown on the lower right corner of the screen.

Once the countdown has reached 0, the app will pause the pump and show you instructions how to finish the pumping session. If you want to continue pumping, press the cancel button and then the START/PAUSE button.

Timer on

Timer off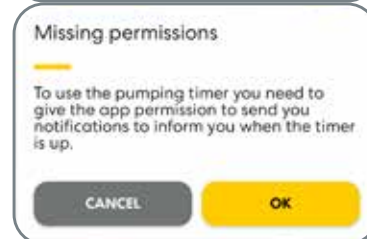

When the timer is active, if you tap the timer icon the app will show a pop-up that allows you to cancel the timer.


i Information


Ensure your smartphone is with you while the app is active, particularly in public environments. If the app is running in the background, keep your smartphone within reach and avoid leaving it unattended.

7. Finishing your pumping session

1. To stop the pumping session, simply tap the FINISH button on the pumping screen. If you're double pumping, you can choose to finish on the Left, Right, or both sides.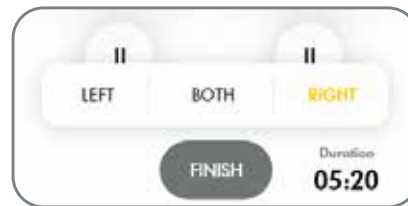


A confirmation pop-up will appear, asking for your confirmation to finish the session.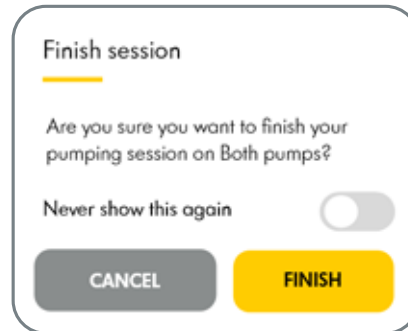


When milk is detected

When milk is not detected


2. Lean forward and break the latch with your finger. Remove the pump from your breast and proceed to finish the session. Hold the pump in a horizontal position while removing it from your breast. **There will be milk in the breast shield tunnel.**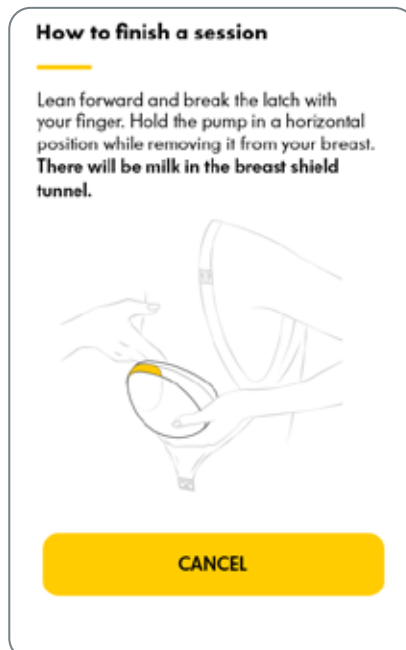

or

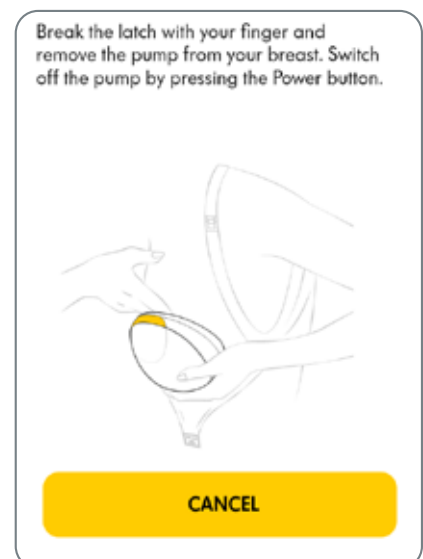3. Place the pump in a **horizontal position**.
The tunnel light will flash and the remaining milk from the breast shield tunnel will drain into the container.
The pump will switch off automatically.

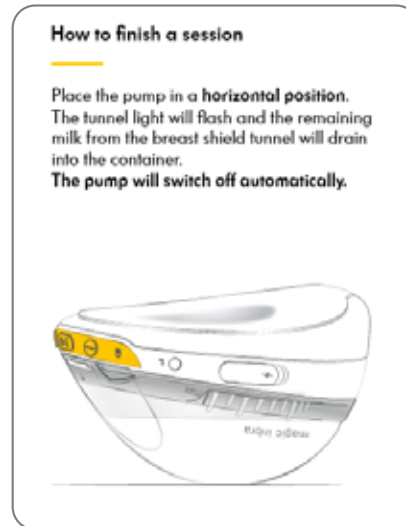
4. The app will show the milk collection status on the screen.
If only one pump has finished, you can continue pumping with the other one.
Once the milk is collected, the pump will switch off.


8. Adjusting the volume and saving a pumping session

Once milk has been collected into the container, the app will display a screen with detailed information about your pumping session. You have the option to manually adjust the pumped milk volume per side. To adjust the volume you can either drag up and down the handles on top of the yellow bars or tap the volume boxes and enter the volume manually with the keypad of the smartphone.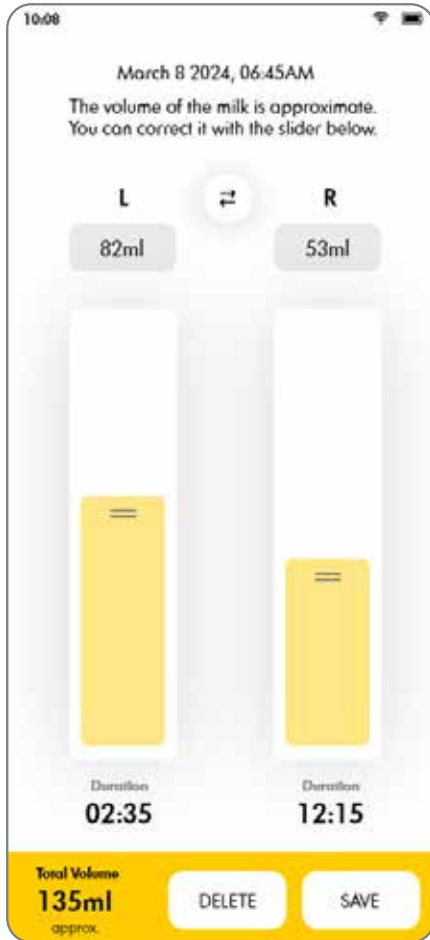

or

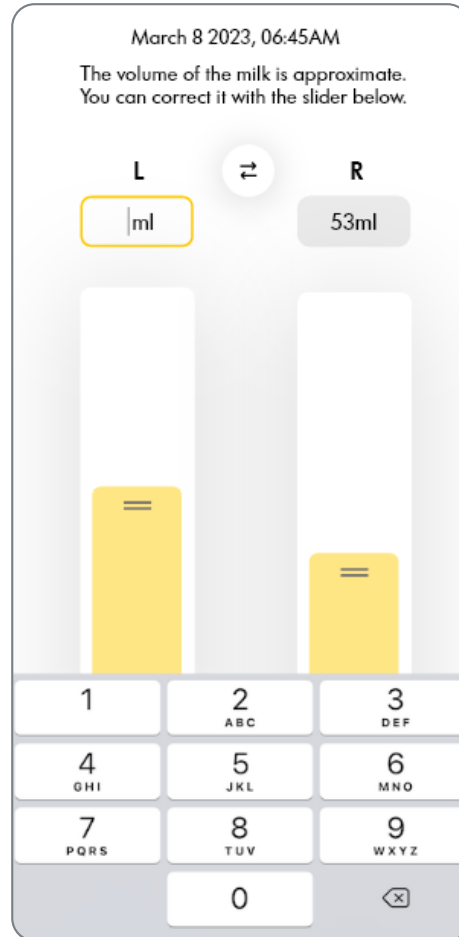
To switch pump side, tap the arrows button in the middle of the screen.

At the bottom of the screen, you will find the buttons to save or delete the session.

When you tap the delete button, a confirmation pop-up will appear.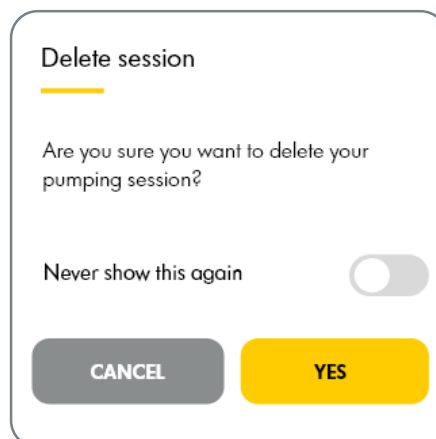

9. Managing and exporting session history

Pumping session data

The app displays information on the active pumping session and saves the session history. You can manually manage session history information and export it for personal use only.


The following information is stored per side in a session:

- Pump name, ID, internal device class & serial number
- Pumping session ID
- Side
- Start date
- Duration
- Milk volume
- Online: Session performed when connected to app
- Offline: Session performed when disconnected from app

To view your saved pumping sessions in chronological order, tap the SESSIONS button on the bottom of the screen.

When you tap a saved session from the list, you can adjust the session information as needed.



Tap the export button on the upper right corner of the screen to export your session data as an excel file using the smartphone's operating system share functionality.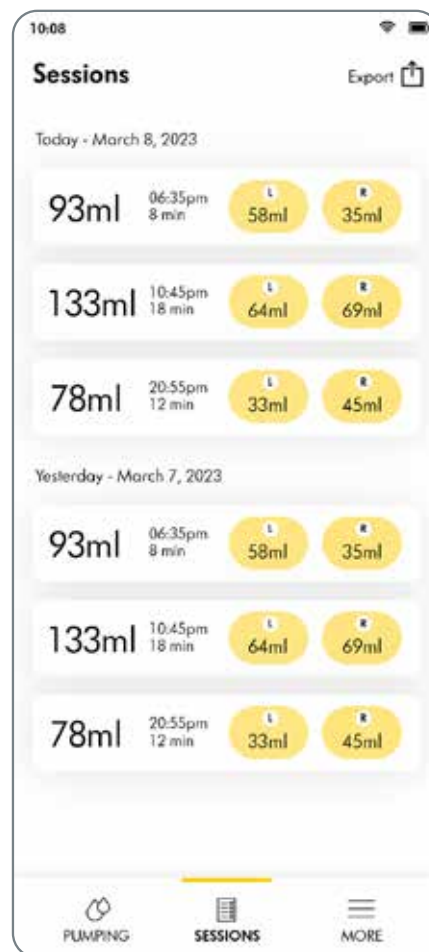


Add a date to sessions with no date available

In some rare cases pumping sessions will be saved without a date and shown on top of the list.


Tap one of the sessions to open it. Then, tap the pencil button.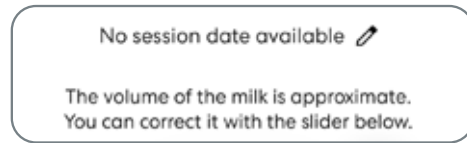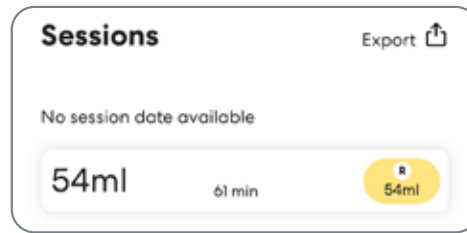

A calendar will open to allow you to correct the date.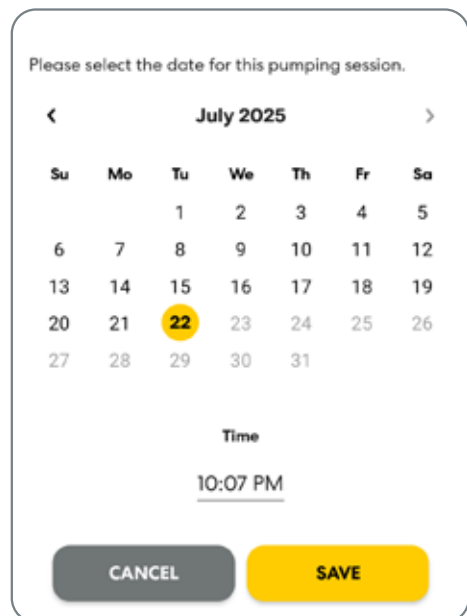


Please make sure you select the correct date, as you will not be able to adjust it later.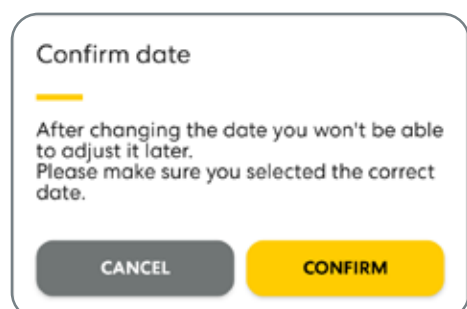

Data Privacy

The app does not collect or store any personal data.

Tapping the export button shows a pop-up that you have to confirm to be able to proceed.

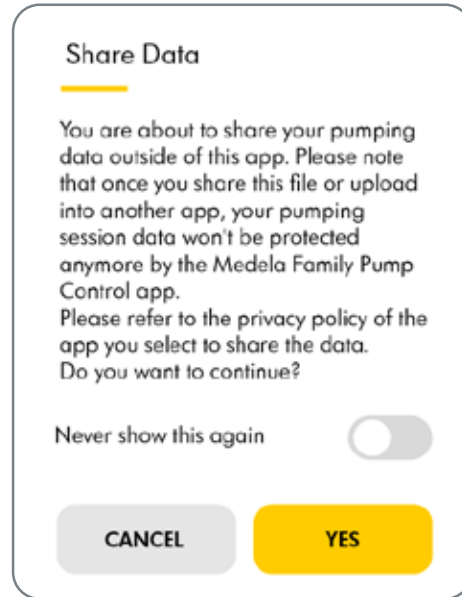iOS:


or

Android:

Offline sessions

The Magic InBra breast pump can save 32 offline sessions. New pumping sessions will replace the older sessions when this limit is reached. Any pumping sessions done without the app will be automatically downloaded and displayed in the app the next time the pump is connected.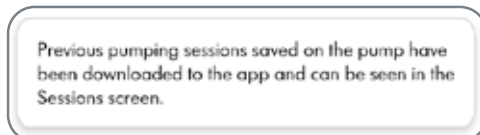

Data exchange between the pump and the app


The data exchanged between the app and the breast pump includes the following:

- Device-to-App: Real-time pump settings (e.g., vacuum strength, session duration), session history (e.g., past pumping session), notifications (e.g. battery low, session complete)
- App-to-Device: User commands (e.g., start/pause pumping, adjust vacuum level).

The app is able to store locally up to 2000 pumping sessions. New pumping sessions will replace the older sessions when this limit is reached.

10. More

On this screen you can find the app’s user manual and Quick start screens, change the app language or measurement units system in App Settings, turn notifications on/off, pair a new breast pump, change your country, find a link to your local customer service and see the legal, regulatory, privacy policy and license information.

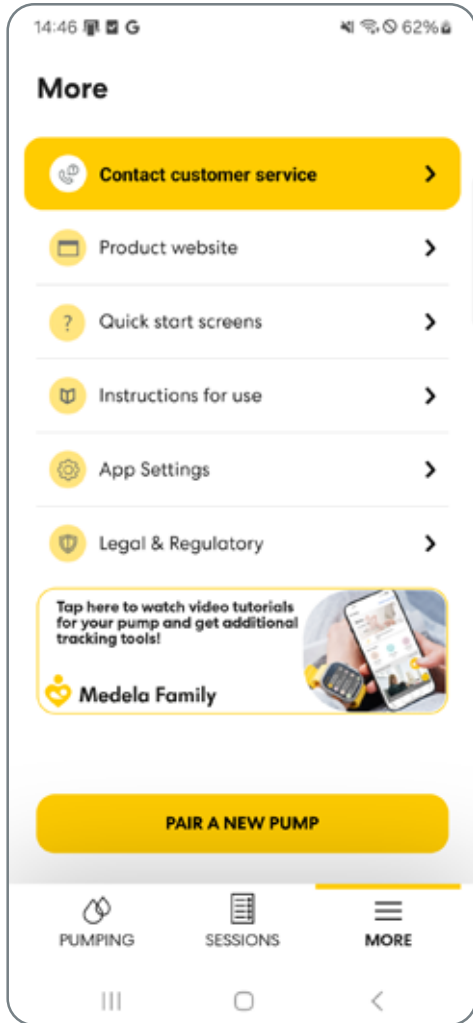End of Support

Medela will provide app support, including security updates, during the app’s lifetime. Users will be notified in advance when app support ends. After this period, updates or security patches may no longer be available.

Software Bill of Materials

Medela provides a Software Bill of Materials (SBOM) under “Legal & Regulatory” > “Components and Licenses”, detailing the software components used in the app.

11. Unpairing the pump

To unpair the pump from your smartphone:

- iOS:
 1. Open the "Settings" app of your smartphone
 2. Navigate to the "Bluetooth" settings
 3. Find the Magic InBra pump from the list of saved devices
 4. Tap the (i) button on the right side and then press the "Forget This Device" button at the bottom of the screen
- Android:
 1. Open the "Settings" app of your smartphone
 2. Navigate to the "Bluetooth" or "Connected devices" settings
 3. Find the Magic InBra pump from the list of saved devices
 4. Tap the "Settings" button and then press the "Forget" or "Unpair" button

To unpair all smartphones from your pump please do the factory reset:

Factory reset

You can perform a factory reset by pressing and holding the Power and Start/Pause buttons for 10 seconds. The factory reset allows you to restore your Magic InBra™ breast pump to its original factory settings if the device is not functioning as expected and/or other troubleshooting steps have failed.

Note: A factory reset will erase all breast pump memory. If the app is being used, all data stored in the app will remain accessible. For further information on data stored in the app, please refer to chapter 9.

If two pumps were paired, please repeat the procedures on the second pump.

12. App uninstall

If you are not using the app, remove the app from your smartphone:

- iOS:
 1. Tap and hold the app icon for a few seconds,
 2. Tap "Remove app"
 3. Tap "Delete app".
- Android: Depending on your Android model, you might need to
 - a) 1. Tap and hold the app icon for a few seconds.
 2. Select "Uninstall".
 - b) 1. Tap and hold the app icon and move it to the top of the screen. Doing this will make an "Uninstall" area appear on top of the screen.
 2. Drop the icon there to remove the app.

Uninstalling the app will erase all app related data stored on the smartphone.

13. Troubleshooting

In case of an unexpected behavior of the app, refer to the troubleshooting table below. If you identify the issue in bold text, then follow the corresponding instructions.

Problem	Solution
<p>Can't start pairing process</p>	<ul style="list-style-type: none"> • Please ensure that Bluetooth on your smartphone is turned on. • The first time the app is started, the operating system of your smartphone will ask you if you want to give the app permission to use Bluetooth (iOS) or Nearby devices (Android). You need to give permission, otherwise you will not be able to pair your breast pump (See chapter 5 – “Necessary permissions”). If you denied the permissions, you can open the settings app of your smartphone, search for Medela Family Pump Control, go to permissions and activate Bluetooth (iOS) or Nearby devices (Android). • Ensure that the Bluetooth LED on the side of the pump is flashing. If it is not flashing, the pairing process can not begin. Please ensure to set the pump into Bluetooth pairing mode following the on-screen instructions described in chapter 5 “Pairing the breast pump”.
<p>The app can not find any pump to pair</p>	<ul style="list-style-type: none"> • Ensure that the Bluetooth LED on the side of the pump is flashing. If it is not flashing, the pairing process can not begin. Please ensure to set the pump into Bluetooth pairing mode following the on-screen instructions described in chapter 5 “Pairing the breast pump”. • Please also ensure that the pump(s) have sufficient battery. If necessary, recharge the pump(s). • Please also ensure that the pump(s) are close (20 cm/8 inches) to the smartphone.
<p>The app can not connect to the pump previously paired or it loses the connection</p>	<ul style="list-style-type: none"> • Ensure that the Bluetooth LED on the side of the pump is permanently on. If it is not on, then the pump is currently not connected. • If necessary, pair the pump again as shown in chapter 5. • Please also ensure that the pump(s) have sufficient battery. If necessary, recharge the pump(s). • Please also ensure that the pump(s) are close (20 cm/8 inches) to the smartphone.
<p>I initially purchased one pump and now I purchased a second pump, how do I pair the second pump?</p>	<p>Tap the “More” button on the bottom of the screen. Tap the “Pair a new pump” button and follow the on-screen instructions to pair your pump.</p>
<p>Can I pair the pump to multiple smartphones</p>	<p>Yes, you can pair the pump with multiple smartphones, but you can only use the pump on one smartphone at the time. Before using it on a second phone, make sure to close the app and turn off Bluetooth on the first smartphone.</p>
<p>I pressed the finish button and removed the pump from my breast but nothing happens</p>	<p>After removing the pump from your breast you need to place the pump on a flat surface in a horizontal position for a few seconds. The pump will collect your milk into the container. Once the milk is collected the pump will switch off. Please refer to chapter 7 “Finishing your pumping session”.</p>

Pump not responding when pressing app buttons	Please ensure that the pump has been correctly paired with the app and that the pump is correctly connected to the app. Please refer to chapter 5 "Pairing the breast pump".
I can't find the Medela Family Pump Control app on the app stores	The Medela Family Pump Control app is only available in countries where the Magic InBra breast pump is sold.* *With the exception of Hong Kong, where the app is available despite Magic InBra™ not being sold.
What happens to my pumping session data if I use two or more smartphones to connect to my pump?	If you use two or more smartphones, pumping sessions will be downloaded to each phone individually, but no data is shared between the smartphones.

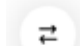
Incident Response

In the event a cybersecurity incident is suspected (e.g. unstable BLE connection, unexpected change of user settings), please contact Medela Customer Service for further assistance.

If you have not resolved the problem with the app, you have further questions or want a printed version of this IFU please contact Medela Customer Service. For contact data visit www.medela.com. Under "Contact" choose your country.

14. Meaning of symbols

The following table explains the meaning of the symbols found in the app.

	Indicates milk status [†]		Indicate the selection of left/right breast for pumping [†]
	Indicates battery status [†]		Indicates that this device contains Bluetooth wireless technology. (Trademarks of Bluetooth Special Interest Group (SIG)) [†]
	Indicates the manufacturer ¹		Maximum Comfort Vacuum level [†]
	Indicate Start/Pause buttons [†]		Stimulation mode [†]
	Indicate increase/decrease vacuum buttons [†]		Expression mode [†]
	Pumping screen [†]		Switch on/off the LED lights of the pump [†]
	Session [†]		Export [†]
	More [†]		Change sides [†]
	Indicates the authorized representative in the European Union ²		Indicates the importer ³

Indicates the date of manufacture ⁴

Indicates the batch code / lot number ⁵

Indicates the item is a medical device ⁶

Indicates the need for the user to consult the instructions for use ⁷

Indicates a carrier that contains unique device identifier information ⁸

Indicates the catalogue number ⁹

Indicates compliance with the EU requirements for medical devices ^{*,†}

References

- ¹ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.1 Manufacturer
- ² ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.2, EC Representative
- ³ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.8, Importer
- ⁴ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.3 Date of manufacture
- ⁵ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.5 Batch code / lot number
- ⁶ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.7.7, Medical Device
- ⁷ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.4.3, Consult Instructions for Use
- ⁸ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.7.10, Unique Device Identifier
- ⁹ ISO 15223-1, Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements, Clause 5.1.6 Catalogue number

* These symbols are not applicable for the US, Canada, Mexico market.

† These symbols are not derived from standards.

Medela AG
Lättichstrasse 4b
6340 Baar, Switzerland
Phone: +41 41 562 51 51
www.medela.com

UK Responsible Person:
Medela UK Ltd.
Unit 3, Huntsman Drive,
Northbank Industrial Park, Irlam,
Manchester M44 5EG,
United Kingdom
www.medela.com/support

Medela Australia Pty Ltd.
3 Arco Lane,
Heatherton, Vic 3202
Australia

Medela Medizintechnik
GmbH & Co. Handels KG
Georg-Kollmannsberger-Str. 2,
85386 Eching, Germany

Distributed by - Distribuido por - Distribué par
Medela LLC
1101 Corporate Drive
McHenry, IL 60050 USA
Phone (800) 435-8316 or (815) 363-1166
customer.service@medela.com
www.medela.com

Trademark recognition:

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Medela AG is under license. Android, Google Play and the Google Play logo are trademarks of Google Inc. iPhone®, Apple® and the Apple logo are trademarks of Apple Inc. Apple Store® is a service mark of Apple Inc.

By downloading the app you agree to the End-User License Agreement of the respective app store.

© Medela AG 2024-2026, ref. master doc 101047055/2026-4/H,
101047441/2026-4/D/EN

REF 101046623

Medela Family Pump Control Smartphone App

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Het trademark en de logo's van Bluetooth® zijn geregistreerde handelsmerken van Bluetooth SIG, Inc. en het gebruik van dergelijke merken door Medela AG is onder licentie. Android, Google Play en het Google Play logo zijn handelsmerken van Google Inc. iPhone®, Apple® en het Apple logo zijn handelsmerken van Apple Inc. Apple Store® is een servicemerk van Apple Inc.
©2024-2024-2026 Medela AG, ref. master doc 101047055/2026-4/H,