

Silicone Suction Cups

Product Information



Reusable Silc cups



Single use Silc cup



CaesAid cup for caesarean sections



Basic suction pump with VAD system

Medela offers a complete system that includes a surgical suction pump with all the necessary components and a large choice of suction cups. The goal is to support the mother and ensure that delivery of the baby can take place in a gentle and controlled manner. The Medela Vacuum assisted delivery (VAD) system provides rapid vacuum-build up in seconds and quick set-up thanks to compatible parts. During a vacuum-assisted delivery, the soft suction cup of the vacuum is applied to the baby's head, and the vacuum helps guide the baby out of the birth canal.

REUSABLE SILICONE CUPS

This medical-grade silicone cup attaches gently to the fetal scalp. Its one-piece design for easy handling and its smooth surface with a longitudinal ridge make it possible to observe rotation. The reusable silicone suction cups are autoclavable and come in 50 mm and 60 mm sizes.

SINGLE USE SILICONE CUPS

The ventouse attaches gently to the fetal scalp. Its smooth surface with a longitudinal ridge allows observing rotation. It is made of thermoplastic elastomer and is 60 mm in diameter.

CAESAID CUPS FOR CAESARIAN SECTIONS

The CaesAid cup offers gentle support for delivery during caesarean sections. The use of the CaesAid may allow for a smaller incision with less blood loss. Additionally, it can be attached to the pump used for suctioning during the operation. The CaesAid is made of medical-grade silicone, is autoclavable and is 60 mm in size.

Intended use

This obstetrical instrument is utilized to assist a clinician in the delivery of an infant during childbirth.

MATERNAL AND FETAL INDICATIONS

Maternal Indications

- Shortening of second stage labour (e.g. maternal exhaustion, reduced maternal effort ie hypertensives, cardiacs, previous caesarean section, poor progress of 2nd stage, malposition of presenting part, deflected head, asynclitism, inadequate expulsive effort)
- Special Cases (e.g. vacuum caesarean section, delivery of second twin)

Fetal indications

- Fetal distress (eg expedite delivery with abnormal CTG, bradycardia)

Conditions

- Gestational age > 36 weeks, estimated fetal weight of 2,500 g or more
- Vertex presentation, with the head well-flexed
- Ruptured membranes
- Cervix fully dilated
- Head fully engaged
- Empty bladder

Maternal Contraindications

- Unengaged fetal head
- Cephalopelvic disproportion (CPD)
- Cervix not fully dilated
- Uncooperative subject
- Failed forceps
- Cup disengaged twice (or more)
- HIV positive status

Fetal Contraindications

- Prematurity < 36 weeks gestation or fetal weight < 2,500 g
- Abnormal presentation
- Fetal abnormalities
- Bleeding disorders/coagulopathies
- Osteogenesis imperfect
- Excessive caput/moulding
- Fetal scalp damage

Obstetric Product Range – Reusable (250 Reprocessing cycles)	Item number
Silc Cup, Ø 50 mm, blue	077.0761
Silc Cup, Ø 60 mm, yellow	077.0762
Silc Cup with trumpet valve, Ø 60 mm	077.0078
CaesAid Cup	077.0080

Single use cups	Item number
Single Use Silc Cup, Ø 60 mm (Polymethylmethacrylat)	077.0791

Tubings, reusable	Item number
Silicone tubing, transparent, 150 cm (for Bird and Silc Cups) Ø 7 x 12 mm	077.0056
Silicone tubing, transparent, 200 cm (for CaesAid Cup)	077.0051

Tubings, disposable	Item number
Disposable tubing PVC, 150 cm, CH25, with two connectors (for reusable Bird Cups)	077.0184
Disposable PVC tubing, sterile, 150 cm for Malmström Cup and Silc Cups	077.0170

Additional reusable cups

Bird Anterior Cup Ø 40 mm	Item number
Consisting of: Chain (without handle), length 11 cm Material: Stainless steel	 077.1041
Bird Anterior Cup Ø 50 mm	Item number
Consisting of: Chain (without handle), length 11 cm Material: Stainless steel	 077.1042
Bird Anterior Cup Ø 60 mm	Item number
Consisting of: Chain (without handle), length 11 cm Material: Stainless steel	 077.1043
Bird Posterior Cup Ø 50 mm	Item number
Consisting of: Chain (without handle), length 11 cm Material: Stainless steel	 077.0063
Steel Handle	Item number
Traction handle with hook to Bird Suction Cups	077.1044

Accessories	Item number
Apgar Timer for standard rail	017.0015
Tubing holder, left side	077.0751
Tubing holder, right side	077.0752
Clampholder, plastic	077.0521

Recommended with Basic suction pump	
Controls	Item number
Foot on/off switch	077.0723
Foot vacuum regulator	077.0731

Recommended with Basic suction pump

Basic suction pump	Item number
Basic suction pump, rack version	071.0001
Trolley	071.0034

Main compatible accessories

Disposable Collection System	Item number
Suction jar PC, 1.5l	077.0082
Disposable suction liner 1.5l	077.0083
Disposable suction liner 1.5l with solidifier	077.0084
Suction jar PC, 2.5l	077.0085
Disposable suction liner 2.5l	077.0086
Disposable suction liner 2.5l with solidifier	077.0087

Filters

	Item number
Disposable virus filter with Medela connection	101035265
Disposable overflow protection/bacteria filter with Medela connection	077.0571
Disposable bacteria filter with Medela connection	077.0573
Disposable bacteria/odour filter with Medela connection	077.0575
Disposable virus filter with overflow protection with Medela connection	101035263

Vacuum tubing, silicone

	Item number
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 25 m	077.0900
Silicone tubing Ø 7 x 12 mm, without coupling pieces 1 m	077.0901
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 2 m	077.0902
Silicone tubing Ø 7 x 12 mm, without coupling pieces, 5 m	077.0905
Silicone tubing Ø 7 x 12 mm with 1 coupling piece, 60 cm	077.0912
Silicone tubing Ø 7 x 12 mm with 1 coupling piece, 100 cm	077.0913
Silicone tubing Ø 7 x 12 mm with 2 coupling pieces, 30 cm	077.0921
Silicone tubing Ø 7 x 12 mm with 2 coupling pieces, 60 cm	077.0922
Silicone tubing Ø 7 x 12 mm, 25 cm, with 1 coupling piece and 1 coupling piece 90°	077.0931
Silicone tubing green Ø 6.5 x 12.5 mm 180m with coupling piece	077.0961
Silicone tubing Ø 7 x 12 mm, 60 cm, with 1 coupling piece and 1 coupling piece 90°	077.0185
Silicone tubing Ø 5 x 10 mm, 25 m	077.0970

Find the full Medela PVS portfolio [here](#)

For more information about Medela products, you can visit www.medela.com

CE 0123

Medical Vacuum Technology for Healthcare Professionals
Please contact us or your local Medela representative for details.

 Medela AG
Lättichstrasse 4b
6340 Baar
Switzerland
www.medela.com

Local contact