

*No. 1 choice
of hospitals
& mothers*

Phototherapy Innovations for the Tiniest



BiliBed®

Phototherapy System

A newborn baby with hyperbilirubinemia needs more than just phototherapy – it particularly needs to be close to its mother.

✓ I'm thrilled! Thanks to the BiliBed my baby could stay with me in my room – despite treatment for jaundice. ✓ Anja B., mother



BiliBed in standard cot

Baby-friendly treatment

Medela places the therapy for the newborn in the cradle, so to speak. Because the BiliBed can be fitted into most standard baby's cots, the baby can remain in its mother's room round the clock. The mother can participate in the care of her baby, thus freeing staff for other duties.

No eye protection necessary

The baby is wrapped in the soft Bilicombi blanket. Because there is no spread light, there is no need for any inconvenient eye protection.

Easy to use

The mattress in the cot is replaced by the BiliBed, and the baby in the fleecy Bilicombi is laid on the baby support. Now the therapy can begin. The nursing staff can control the length of therapy and the total time in operation using the two hour meters.

Highly effective

The BiliBed is equipped with a blue fluorescent tube. The precisely directed light and the minimum distance from the baby ensure high light efficiency and outstanding therapeutic action.

Low operating costs

The BiliBed is an exceptionally economical therapeutical device. It requires only a single fluorescent tube with an extremely long life of 1500 hours. Its power requirements of only 30 W are a further contribution to keeping operating costs to an absolute minimum.

For the baby's well-being

The baby lies in the Bilicombi on a comfortable film mattress. Clinical tests prove that the baby feels extremely comfortable on it.

Studies suggest that intimate mom-baby interaction, including eye contact, is essential for the successful development of newborns*.

*«The Behavioral Dynamics of Mutual Responsiveness in Early Face-to-Face Mother-Infant Interactions,» Douglas K. Symons and Greg Moran, Child Development; «Mother-Infant Bonding,» H.B. Valman, British Medical Journal; «Helping Mothers to Love Their Babies,» British Medical Journal

BiliBed®

Innovation for the Benefit of Mother and Child



BILICOMBI

The washable soft Bilicombi blanket keeps the baby in the correct position.


BABY SUPPORT

The baby lies on a light-permeable, skin-friendly film mattress.

LIGHT UNIT

Positioned in the cot instead of a mattress.

Technical Data

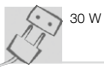

30 W



1 compact fluorescent tube, blue light



E_{bi} max (400-550nm) 49 W/m²



Tube life:
approx. 1500 hours



Total h: time in operation, not resettable
h: therapy time, resettable



L x W x H:
630 x 326 x 134 mm



4 kg



CLASS IPX4



EN 60601-1:1990
EN 60601-2-50:2002

EASY TO USE

- ✓ Baby-friendly therapy in the baby's cot
- ✓ Baby needs no eye protection
- ✓ Two hour meters for the length of therapy and operating time

GENTLE THERAPY

- ✓ Cosy baby support
- ✓ Special fleecy Bilicombi blanket

ECONOMICAL

- ✓ Highly effective
- ✓ Low upkeep costs
- ✓ Energy-saving operation

International Sales

Medela AG,
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 (0)41 769 51 51
Fax +41 (0)41 769 51 00
ism@medela.ch
www.medela.com

Australia

Medela Pty Ltd, Medical Technology
3 Arco Lane, Heatherton
Vic 3202, Australia
Phone +61 (0) 3 9552 8600
Fax +61 (0) 3 9552 8699
contact@medela.com.au
www.medela.com.au

Canada

Medela Inc.
4160 Sladeview Crescent
Unit # 8
Mississauga, Ontario
Canada, L5L 0A1
Phone +1 905 608 7272
Fax +1 905 608 8720
info@medela.ca
www.medela.ca

China

Medela (Beijing)
Medical Technology Co., Ltd.
Unit M, 4F, No 1068 Wuzhong Road,
Minhang District
Shanghai 201103
China
Phone +86 21 62368102
Fax +86 21 62368103
info@medela.cn
www.medela.cn

France

Medela France Sarl
14, rue de la Butte Cordière
91154 Etampes cedex
Phone +33 (0)1 69 16 10 30
Fax +33 (0)1 69 16 10 32
info@medela.fr
www.medela.fr

Germany & Austria

Medela Medizintechnik
GmbH & Co. Handels KG
Postfach 1148
85378 Eching
Phone +49 (0)89 31 97 59-0
Fax +49 (0)89 31 97 59 99
info@medela.de
www.medela.de

Italy

Medela Italia Srl
Via Turrini, 19 – Loc. Bargellino
40012 Calderara di Reno (BO)
Phone +39 051 72 76 88
Fax +39 051 72 76 89
info@medela.it
www.medela.it

Japan

Medela K.K.
KDX Kiba Building 3F
5-12-8, Kiba, Koto-ku,
Tokyo 135-0042 Japan
Phone +81 3 3820-5187
Fax +81 3 3820-5181
info@medela.jp
www.medela.jp

Netherlands & Belgium

Medela Benelux BV
De Steenbok 12
5215 ME 's-Hertogenbosch
Phone +31 73 690 40 40
Fax +31 73 690 40 44
info@medela.nl
info@medela.be
www.medela.nl
www.medela.be

Russia

OOO Medela
Vavilova str. 97, entrance 3
117335 Moscow
Russia
Phone +7 (495) 980 6194
Fax +7 (495) 980 6194
info@medela-russia.ru
www.medela-russia.ru

Spain & Portugal

Productos Medicinales Medela, S.L.
c/ Manuel Fernández Márquez, 49
08918 Badalona (Barcelona)
Spain
Phone +34 93 320 59 69
Fax +34 93 320 55 31
info@medela.es
www.medela.es
Portugal
Phone: 808 203 238
info@medela.pt
www.medela.pt

Sweden

Medela Medical AB
Box 7266
187 14 Täby
Phone +46 (0)8 630 09 40
Fax +46 (0)8 630 09 48
info@medela.se
www.medela.se

Switzerland

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 (0)848 633 352
Fax +41 (0)41 769 51 00
contact@medela.ch
www.medela.ch

United Kingdom

Medela UK Ltd.
Huntsman Drive
Northbank Industrial Park
Irlam, Manchester M44 5EG
UK
Phone +44 (0)161 776 0400
Fax +44 (0) 161 776 0444
info@medela.co.uk
www.medela.co.uk

USA

Medela, Inc.
P.O. Box 660
1101 Corporate Drive
McHenry, IL. 60050
USA
Phone +1 877 735 1626
Fax +1 815 363 2487
customer.service@medela.com
www.medelabreastfeedingus.com